

Invacare® PreciseR_x™ Pediatric Flowmeter

IRCPF16AW

en User Manual



This manual **MUST** be given to the user of the product.
BEFORE using this product, read this manual and save for future reference.



Yes, you can.®

©2017 Invacare Corporation

All rights reserved. Reproduction, duplication or modification in whole or in part is prohibited without prior written permission from Invacare. Trademarks are identified by ™ and ®. All trademarks are owned by or licensed to Invacare Corporation or its subsidiaries unless otherwise noted.

Making Life's Experiences Possible is a registered trademark in the U.S.A.

Contents

1 General	4
1.1 Symbols	4
1.2 Intended Use	5
1.3 Indications for Use	5
1.4 Contraindications.	5
1.5 Technical Description.....	5
2 Safety	7
2.1 General Guidelines	7
3 Setup	11
3.1 Handling	11
3.2 Typical Setup.....	11
3.3 Setup Without a Humidifier Bottle.....	13
3.4 Setup With a Humidifier Bottle	14
3.5 Setting the Flowrate on the Pediatric Flowmeter	15
4 After Use	16
4.1 Recycling Information	16
5 Technical data	17
5.1 Specifications	17
6 Warranty	18
6.1 Limited Warranty	18

1 General

1.1 Symbols



DANGER!

– Danger indicates an imminently hazardous situation which, if not avoided, will result in death or serious injury.



WARNING!

– Warning indicates a potentially hazardous situation which, if not avoided, could result in death or serious injury.



CAUTION!

– Caution indicates a potentially hazardous situation which, if not avoided, may result in property damage or minor injury or both.







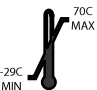



IMPORTANT!

– IMPORTANT indicates a hazardous situation that could result in damage to property if it is not avoided.



Gives useful tips, recommendations and information for efficient, trouble-free use.

	<p>General Warning Sign</p> <p> The background color inside the triangle is yellow on product labels.</p>
	<p>Read Manual</p> <p> The color of the symbol background is blue on product labels.</p>
	<p>No Smoking</p> <p> The color of the circle with diagonal bar is red on product labels.</p>
	<p>No Open Flame</p> <p> The color of the circle with diagonal bar is red on product labels.</p>
	<p>European Community Representative</p>
	<p>Manufacturer</p>
	<p>Date of Manufacture</p>
	<p>Recycle</p>
	<p>Reference Number</p>

	Serial Number
	Indoor Use ONLY
	DO NOT use oil or grease
	Keep dry
	Transport and Storage Temperature
	Input Fitting
	Output Fitting
	This product complies with Directive 93/42/EEC concerning medical devices. The launch date of this product is stated in the CE declaration of conformity.

1.2 Intended Use

The Precise Rx™ Pediatric Flowmeter is intended to provide supplemental oxygen to patients with low-flow prescriptions in a non-acute setting such as the home or long-term care facility. The device is not intended to be life supporting or life sustaining.

1150712-D



WARNING!

Risk of Injury or Damage

Use of this product outside of the intended use and specifications has not been tested and may lead to product damage, loss of product function, or personal injury.

- DO NOT use this product in any way other than described in the specifications and intended use sections of this manual.

1.3 Indications for Use

To provide supplemental oxygen to patients with low-flow prescriptions in a non-acute setting.

1.4 Contraindications

There are no known contraindications.

1.5 Technical Description

The device is a free standing, non-electrically powered, accessory for connection to the output fitting of an Invacare 5 liter output oxygen concentrator that delivers low flow rates of less than one (1) liter per minute to a pediatric patient through a pediatric nasal cannula.

Input oxygen gas at an input pressure of 5.0 +/- 10% PSI and at a minimum flow rate of 2 liters per minute by a compatible Invacare 5 liter output oxygen concentrator is delivered to the device input through an external extension tubing accessory. The input gas is routed to an internal bleed orifice for maintaining a minimum flow output load to the oxygen concentrator for maintaining adequate oxygen concentration values of the delivered gas by the

oxygen concentrator. The input gas is then supplied to the input of a variable flow rate device and flow meter for flow rate adjustment and subsequent delivery to the output fitting for delivery to a pediatric patient through an externally connected administration accessory, a pediatric nasal cannula.

Oxygen gas output can be delivered to the patient in adjustable flow rates in the continuous flow mode over the range of 1/16 to 3/4 liters per minute.

An externally mounted water humidifier can be used in-line with the output fitting to provide humidification of the oxygen gas delivered to the pediatric patient.

2 Safety

2.1 General Guidelines

In order to ensure the safe installation, assembly and operation of the device these instructions **MUST** be followed.

**DANGER!****Risk of Death, Injury or Damage from Fire**

Textiles, oil or petroleum substances, grease, greasy substances and other combustibles are easily ignited and burn with great intensity in oxygen enriched air and when in contact with oxygen under pressure. Smoking during oxygen therapy is dangerous and is likely to result in burns or death. To avoid fire, death, injury or damage:

- DO NOT SMOKE while using this device. DO NOT use near OPEN FLAME or IGNITION SOURCES.
- NO SMOKING signs should be prominently displayed.
- Keep all open flames, matches, lighted cigarettes, electronic cigarettes, or other sources of ignition at least 3 m (10 ft) away from any concentrator or any oxygen carrying accessories such as cannulas or tanks.

**DANGER!****Risk of Death, Injury or Damage from Fire**

Textiles, oil or petroleum substances, grease, greasy substances and other combustibles are easily ignited and burn with great intensity in oxygen enriched air and when in contact with oxygen under pressure. To avoid fire, death, injury or damage:

- Avoid creation of any spark near oxygen equipment. This includes sparks from static electricity created by any type of friction.
- Use only oxygen compatible water-based lotions or salves before and during oxygen therapy. To verify, refer to the lotion/salve container for oxygen compatible water-based statement. If necessary, contact the manufacturer. DO NOT use any lubricants on concentrator or accessory fittings unless recommended by Invacare.
- Keep the oxygen tubing, cord, AC adapter, and concentrator out from under such items as blankets, bed coverings, chair cushions, clothing, and away from heated or hot surfaces including space heaters, stoves, and similar electrical appliances.
- Make sure concentrator is off when not in use.



DANGER!

Risk of Death, Injury or Damage

Improper use of the product may cause death, injury or damage. This section contains important information for the safe operation and use of this product.

- DO NOT use this product or any available optional equipment without first completely reading and understanding these instructions and any additional instructional material such as user manuals, service manuals or instruction sheets supplied with this product or optional equipment.
- If you are unable to understand the warnings, cautions or instructions, contact a healthcare professional, dealer or technical personnel before attempting to use this equipment.
- Check ALL external components and carton for damage. In case of damage, or if the product is not working correctly, contact a technician or Invacare for repair.
- This product is intended to be set up by adults or under adult supervision only after reading and understanding the instructions and warnings of this user manual.
- THE INFORMATION IN THIS DOCUMENT IS SUBJECT TO CHANGE WITHOUT NOTICE.



DANGER!

Risk of Injury or Death

To avoid choking or ingestion of chemicals from airway contamination:

- DO NOT use the concentrator in the presence of pollutants, smoke, fumes, flammable anesthetics, cleaning agents, or chemical vapors.



WARNING!

Risk of Injury or Damage

To prevent injury or damage from misuse:

- NEVER leave concentrator unattended when plugged in.
- Make sure concentrator is off when not in use.



WARNING!

Risk of Injury or Damage

– Invacare products are specifically designed and manufactured for use in conjunction with Invacare accessories. Accessories designed by other manufacturers have not been tested by Invacare and are not recommended for use with Invacare products.

- No modification of this equipment is allowed.

**WARNING!****Risk of Injury**

A change in altitude may affect total oxygen available for use. To prevent oxygen deprivation:

- Consult a physician before traveling to higher or lower altitudes to determine if the patient's flow settings should be changed.

**WARNING!****Risk of Injury or Death**

To prevent injury or death from product misuse:

- Closely supervise when this device is used by or near children or physically-challenged individuals.
- Monitor patients using this device who are unable to hear or see alarms or communicate discomfort.
- Keep out of reach of children and away from pets.

**DANGER!****Risk of Injury or Death**

While Invacare strives to produce the best oxygen devices in the market today, the oxygen concentrator can fail to produce oxygen due to power failure or device malfunction.

- Contact your provider for a backup source of oxygen for a pediatric patient.
- Do not connect the input of the device to any other oxygen delivery device other than an Invacare 5 liter output oxygen concentrator, otherwise serious injury can occur.
- The Pediatric Flowmeter is designed for an input pressure of 34.5 kPa (5 psi) only. The flowmeter is applicable only with Invacare concentrators with 34.5 kPa (5 psi) output pressure.

**WARNING!****Risk of Injury or Damage**

To prevent injury or damage during use:

- For optimum performance, Invacare recommends that the concentrator be on and running for a minimum of 30 minutes. Shorter periods of operation may reduce maximum product life.
- For proper oxygen output, the concentrator should be used in an upright position.
- The Pediatric Flowmeter cannot be used in conjunction with PAP, Bi-Level, mechanical ventilator or other such devices.



WARNING!

Risk of Injury or Damage

The Pediatric Flowmeter is specifically designed to eliminate routine preventive maintenance. To prevent injury or damage:

- Only professionals of the healthcare field or persons fully conversant with this process such as factory trained personnel should perform service, except for tasks described in this manual.
- Users should contact your dealer or Invacare for service.
- DO NOT disassemble. Refer servicing to qualified service personnel. There are no user serviceable parts.



CAUTION!

Risk of Damage


To prevent damage from liquid ingress:

- If the device is not working properly, if it has been dropped or damaged, or dropped into water, call equipment provider/qualified technician for examination and repair.
- NEVER drop or insert any object or liquid into any opening.
- Use of this device with a humidifier located on the inlet gas stream may impair performance or damage the Pediatric Flowmeter. Use of a humidifier is limited to use only on the output gas being delivered by the Pediatric Flowmeter to the patient. Refer to Setup With a Humidifier Bottle.

3 Setup

3.1 Handling

Unpacking and Inspection

1. Check for any obvious damage to the carton or its contents. If damage is evident, notify your carrier/dealer for further instruction.
2. Remove all the loose packing from the carton.
3. Carefully remove all components from the carton.
 -  Packing materials and carton should be kept for future use during storage or transportation.
4. Examine exterior of the flowmeter for nicks, dents, scratches or other damages. Inspect all components.

Storage

1. Store the repackaged flowmeter in a dry area.
2. DO NOT place other objects on top of the repackaged device.

Cleaning

The unit exterior can be cleaned with warm soapy water. Wipe with a soft, damp cloth and dry thoroughly.

3.2 Typical Setup



WARNING!

- ONLY use a pediatric nasal cannula with this device.
- Place the pediatric nasal cannula on patient only after all tubing connections have been made. Possible water feedback through the tubing could occur in this sequence is not followed.
- When adjusting the Pediatric Flowmeter, the flowmeter on the oxygen concentrator MUST be set at 2.0 L/min. Failure to comply with this procedure will cause incorrect flow rates to the patient.
- Invacare recommends that crush resistant oxygen tubing (not supplied) be used with this product.



WARNING!

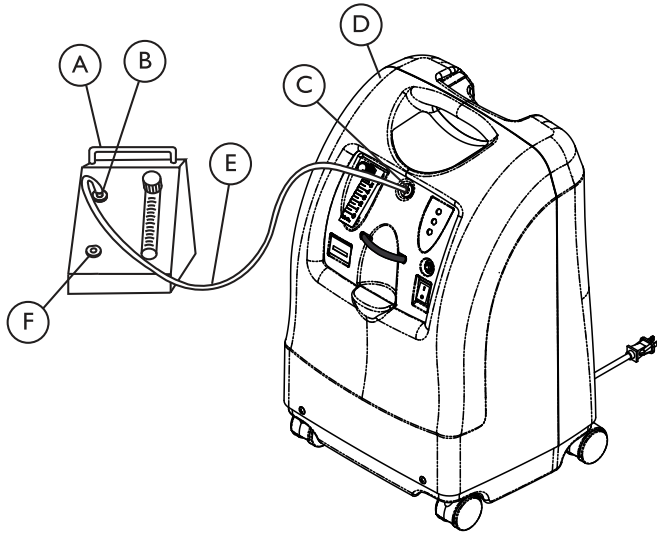
- If use of a humidifier bottle is not prescribed, the oxygen tubing is not to exceed 7.6 m (25 ft) in total length from the Pediatric Flowmeter to the patient, and the total length of ALL tubing CANNOT exceed 15.2 m (50 ft).
- If use of a humidifier bottle is prescribed, the oxygen tubing is not to exceed 7.6 m (25 ft) in total length from the humidifier bottle to the patient, and the total length of ALL tubing for the Pediatric Flowmeter CANNOT exceed 11.8 m (39 ft).

1. Turn the Invacare 5 liter output concentrator on. An indicator light on the control panel will illuminate. Refer to the concentrator user manual for setup and use.
2. Set the oxygen concentrator to 2.0 L/min.
3. Place the Pediatric Flowmeter stand on table top.
4. Turn the Pediatric Flowmeter knob counterclockwise to the fully open position.
5. Allow concentrator to run a minimum of 30 minutes to ensure proper concentration level. Check and readjust the oxygen concentrator to 2.0 L/min.

Accessories – Oxygen tubing, tubing connector, pediatric nasal cannula, and humidifier bottle with adapter are not supplied with the Pediatric Flowmeter.

- Optional accessories: Standard Humidifier — Invacare Part Number 1155719.

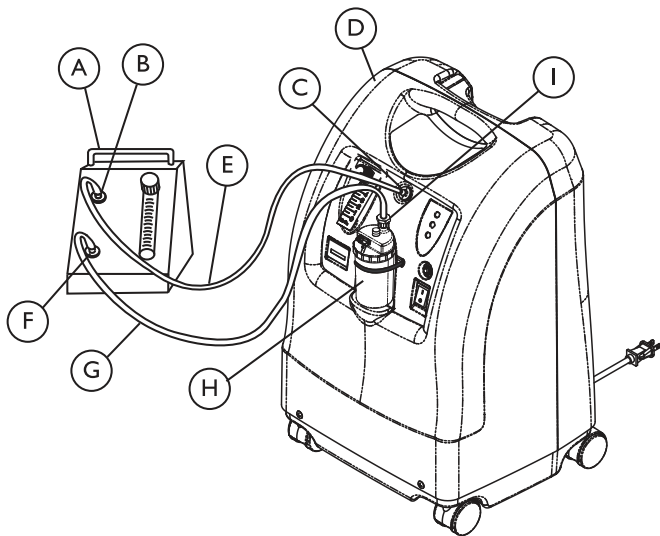
3.3 Setup Without a Humidifier Bottle



1. Connect a 2.1 m (7 ft) oxygen tubing (E) from the output fitting (C) of the concentrator (D) to the input fitting (B) of the Pediatric Flowmeter (A).
2. Connect a 7.6 m (25 ft) maximum length oxygen tubing (not shown) to the output fitting (F) of the Pediatric Flowmeter (A).
3. Attach the opposite end of the oxygen tubing (not shown) to a pediatric nasal cannula using an appropriate oxygen tubing connector as applicable (not shown).

(A)	Pediatric Flowmeter
(B)	Input Fitting on Pediatric Flowmeter
(C)	Output Fitting of Concentrator
(D)	Concentrator
(E)	2.1 m (7 ft) Oxygen Tubing
(F)	Output Fitting of Flowmeter

3.4 Setup With a Humidifier Bottle



(A)	Pediatric Flowmeter
(B)	Input Fitting of Pediatric Flowmeter
(C)	Output Fitting of Concentrator
(D)	Concentrator
(E)	2.1 m (7 ft) Oxygen Tubing from Concentrator to Flowmeter
(F)	Output Fitting of Flowmeter

(G)	2.1 m (7 ft) Oxygen Tubing from Flowmeter to Concentrator
(H)	Humidifier Bottle
(I)	Inlet of Humidifier Bottle

1. Connect a 2.1 m (7 ft) maximum oxygen tubing (E) from the output fitting (C) of the concentrator (D) to the input fitting (B) of the Pediatric Flowmeter (A).
2. Follow the directions supplied by the humidifier bottle (H) manufacturer for instructions on filling and connecting to oxygen tubing.
3. After filling the humidifier bottle with water, place it in the humidifier compartment on the Invacare concentrator (D) (see concentrator user manual for mounting the bottle).
4. Connect 2.1 m (7 ft) maximum oxygen tubing (G) from the Pediatric Flowmeter (A) output fitting (F) to the inlet of the humidifier bottle (I) (using an appropriate adapter if applicable).
5. Connect a 7.6 m (25 ft) maximum oxygen tubing to the outlet of the humidifier bottle (not shown).
6. Attach the opposite end of the oxygen tubing to a pediatric nasal cannula using an appropriate oxygen tubing connector (not shown).



CAUTION!

– DO NOT coil crush resistant oxygen tubing on COLD floor or condensation will form inside the tubing.

3.5 Setting the Flowrate on the Pediatric Flowmeter



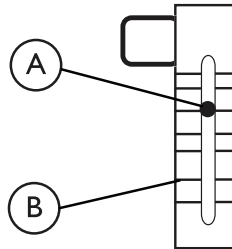
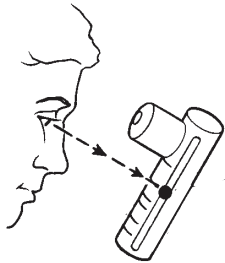
WARNING!



– DO NOT select a different setting unless the patient's physician has prescribed a change in the flowrate of the Pediatric Flowmeter.



CAUTION!

– DO NOT allow water or other substances to enter flowmeter openings. DO NOT submerge the flow stand, flowmeter or tubing in water.



1. Set the Pediatric Flowmeter as prescribed by the patient's physician.
 -  The flowrate on the standard concentrator flowmeter will drop below the 2.0 L/min setting after the flowmeter is installed and operating. The Pediatric Flowmeter setting should be reassessed periodically.
2. When adjusting the flow, center the ball **A** of the Pediatric Flowmeter **B** with the line of the prescribed flowrate. When viewing from the proper angle, the graduation lines on the front and the back of the flowmeter will be in alignment with the center of the ball.
3. Check for oxygen delivery by placing the nasal prongs of the pediatric nasal cannula under the surface of a half-full glass of water only when the oxygen is flowing and look for bubbles indicating proper operation. Wipe any excess water off the nasal prongs when finished before placing on the patient.
4. Place pediatric nasal cannula on the patient.
 -  The proper placement and positioning of the prongs of the pediatric nasal cannula in the nose is critical to the amount of oxygen delivered to the respiratory system of the patient.

4 After Use

4.1 Recycling Information

This product has been supplied from an environmentally aware manufacturer.

Follow local governing ordinances and recycling plans regarding disposal of the device or accessories normally used in operation. The device does not generate waste or residue in operation.

Any accessories not part of the device **MUST** be handled in accordance with the individual product marking for disposal.

5 Technical data

5.1 Specifications

INPUT PRESSURE RATING:	34.5 ± 3.45 kPa (5 ± 0.5 psi)
OXYGEN INPUT:	2 L/min, 85% to 95.6% oxygen
MAXIMUM OUTPUT PRESSURE:	27.6 kPa (4.0 psi)
OXYGEN OUTPUT FLOW:	0–750 cc/min (0–3/4 L/min)
FLOW ADJUSTABLE IN INCREMENTS:	50 cc/min (1/16 L/min)
FLOW ACCURACY:	± 10% of full scale
WIDTH:	11.94 cm (4.7 in)
HEIGHT:	14.48 cm (5.7 in)
DEPTH:	14.48 cm (5.7 in)
WEIGHT:	0.8 kg (1.86 lb)
SHIPPING WEIGHT:	1.1 kg (2.40 lb)
OPERATING TEMPERATURE RANGE:	10° to 35° C (50° to 95° F)
STORAGE TEMPERATURE RANGE:	–29° to 70° C (–20° to 158° F)



Measurement uncertainty is included in the device specification. All conditions at ambient temperature and pressure, dry.

6 Warranty

6.1 Limited Warranty

Terms and conditions of the warranty are part of the general terms and conditions particular to the individual countries in which this product is sold.

Contact information for your local Invacare office is located inside the back cover of this manual.

Belgium & Luxemburg: Invacare nv, Autobaan 22, B-8210 Loppem • Tel: (32) (0) 50 83 10 10 • Fax: (32) (0) 50 83 10 11 • belgium@invacare.com • www.invacare.be

Danmark: Invacare A/S, Sdr. Ringvej 37, DK-2605 Brøndby • Tel: (45) (0)36 90 00 00 • Fax: (45) (0)36 90 00 01 • denmark@invacare.com • www.invacare.dk

Deutschland: Invacare GmbH, Alemannenstraße 10, D-88316 Isny • Tel: (49) (0)75 62 7 00 0 • Fax: (49) (0)75 62 7 00 66 • kontakt@invacare.com • www.invacare.de

Eastern Europe, Middle East & CIS: Invacare EU Export • Kleiststraße 49 • D-32457 Porta Westfalica • Germany • Tel: (49) 5731 754540 • Fax: (49) 5731 754541 • webinfo-eu-export@invacare.com • www.invacare-eu-export.com

España: Invacare SA, c/Areny s/n, Polígon Industrial de Celrà, E-17460 Celrà (Girona) • Tel: (34) (0)972 49 32 00 • Fax: (34) (0)972 49 32 20 • contactsp@invacare.com • www.invacare.es

France: Invacare Poirier SAS, Route de St Roch, F-37230 Fondettes • Tel: (33) (0)2 47 62 64 66 • Fax: (33) (0)2 47 42 12 24 • contactfr@invacare.com • www.invacare.fr

Ireland: Invacare Ireland Ltd, Unit 5 Seatown Business Campus • Seatown Road, Swords, County Dublin – Ireland • Tel: (353) 1 810 7084 • Fax: (353) 1 810 7085 • ireland@invacare.com • www.invacare.ie

Italia: Invacare Mecc San s.r.l., Via dei Pini 62, I-36016 Thiene (VI) • Tel: (39) 0445 38 00 59 • Fax: (39) 0445 38 00 34 • italia@invacare.com • www.invacare.it

Nederland: Invacare AE, Galvanistraat 14–3, NL–6716 BZ Ede • Tel: (31) (0)318 695 757 • Fax: (31) (0)318 695 758 • nederland@invacare.com • csede@invacare.com • www.invacare.nl

Norge: Invacare AS, Grensesvingen 9, Postboks 6230, Etterstad, N-0603 Oslo • Tel: (47) (0)22 57 95 00 • Fax: (47) (0)22 57 95 01 • norway@invacare.com • island@invacare.com • www.invacare.no

Österreich: Invacare Austria GmbH, Herzog Odilostrasse 101, A-5310 Mondsee • Tel: (43) 6232 5535 0 • Fax: (43) 6232 5535 4 • info@invacare-austria.com • www.invacare.at

Portugal: Invacare Lda • Rua Estrada Velha, 949, P-4465-784 Leça do Balio • Tel: (351) (0)225 1059 46/47 • Fax: (351) (0)225 1057 39 • portugal@invacare.com • www.invacare.pt

Sverige: Invacare AB • Fagerstagatan 9 • S-163 53 Spånga • Tel: (46) (0)8 761 70 90 • Fax: (46) (0)8 761 81 08 • sweden@invacare.com • www.invacare.se

Suomi: Camp Mobility • Patamäenkatu 5, 33900 Tampere • Tel: 09-350 76 310 • info@campmobility.fi • www.campmobility.fi

Schweiz/Suisse/Svizzera: Invacare AG • Benkenstrasse 260 • CH-4108 Witterswil • Tel.: (41) (0)61 487 70 80 • Fax.: (41) (0)61 487 70 81 • switzerland@invacare.com • www.invacare.ch

United Kingdom: Invacare Limited, Pencoed Technology Park, Pencoed, Bridgend CF35 5HZ • Tel: (44) (0) 1656 776222 • Fax: (44) (0) 1656 776220 • UK@invacare.com • www.invacare.co.uk

Invacare Corporation

USA

One Invacare Way
Elyria, Ohio 44035
United States
Tel: 440-329-6000
Tel: 800-333-6900
Technical Services
Tel: 440-329-6593
Tel: 800-832-4707

www.invacare.com



EU Representative

Invacare Deutschland GmbH
Kleiststraße 49
D-32457 Porta Westfalica
Germany
Tel: (49) (0) 5731 754 0
Fax: (49) (0) 5731 754 52191



Manufacturer

Invacare Corporation
2101 E. Lake Mary Blvd.
Sanford, FL 32773
United States
Tel: 407-321-5630



1150712-D 2017-05-05



Making Life's Experiences Possible®



Yes, you can.®