Invacare® Delta™

en	Rollator User Manual	3
de	Rollator Gebrauchsanweisung.....	17
es	Rollator Manual del usuario.....	31
fr	Déambulateur Manuel d'utilisation	45
it	Rollator Manuale d'uso.....	59
nl	Rollator Gebruiksaanwijzing	73
pt	Andarilho Manual de utilização	87

This manual **MUST** be given to the user of the product.
BEFORE using this product, read this manual and save for future reference.

Yes, you can.®

© 2016 Invacare® Corporation

All rights reserved. Republication, duplication or modification in whole or in part is prohibited without prior written permission from Invacare. Trademarks are identified by ™ and ®. All trademarks are owned by or licensed to Invacare Corporation or its subsidiaries unless otherwise noted.

Contents

This manual **MUST** be given to the user of the product.
BEFORE using this product, read this manual and save for future reference.

1 General	4
1.1 About this manual	4
1.2 Intended use	4
1.3 Service Life	4
1.4 Scope of delivery	5
1.5 Warranty	5
2 Safety	6
2.1 General warnings	6
2.2 Labels and symbols on the product	6
3 Usage	7
3.1 Before use	7
3.2 Operating position	7
3.3 Folding and unfolding the rollator	8
3.4 Mounting the bag, basket and tray	9
3.5 Using the hand brakes	9
3.6 Adjusting the handle height/angle	10
3.7 Walking	11
3.8 Sitting down on a chair	11
3.9 Rising to a standing position	11
4 Maintenance and care	13
4.1 Maintenance	13
4.2 Cleaning	13
5 After Use	14
5.1 Disposal	14
5.2 Re-use	14

6 Technical data	15
6.1 Dimensions and weight	15
6.2 Material	15

I General

I.1 About this manual

This User Manual contains important information about your product. In order to ensure safety when using the product, read the User Manual carefully and follow the safety instructions.

The person who prescribes the product or who is otherwise responsible for the user must ensure that the product is suitable for the user. The user shall be instructed how to use the product. This includes the use of the features and the restrictions in use. The user shall also be notified of the warnings for use given in the user manual.

Symbols in this manual

In this User Manual warnings are indicated by symbols. The warning symbols are accompanied by a heading that indicates the severity of the danger.

WARNING

Indicates a hazardous situation that could result in serious injury or death if it is not avoided.

CAUTION

Indicates a hazardous situation that could result in minor or slight injury if it is not avoided.

IMPORTANT

Indicates a hazardous situation that could result in damage to property if it is not avoided.

Tips and Recommendations

Gives useful tips, recommendations and information for efficient, trouble-free use.

This product complies with Directive 93/42/EEC concerning medical devices. The launch date of this product is stated in the CE declaration of conformity.

I.2 Intended use

Indications

The rollator is intended for support and improvement of a restricted walking ability. The user's remaining walking ability needs to be sufficient for leaving the home, for usual daily activities and for participating in social life without endangering themselves or others.


The rollator is intended to support a person during walking. Furthermore the rollator is intended to carry a load either in a specially designed basket, on a tray or in a bag.

The max. user weight is 135 kg. This weight limitation includes the weight of the user and the content of the basket.

I.3 Service Life

The expected service life of this product is five years when used in accordance with safety instructions, maintenance intervals and correct use, stated in this manual. The effective product life cycle can vary according to frequency and intensity of use.

I.4 Scope of delivery

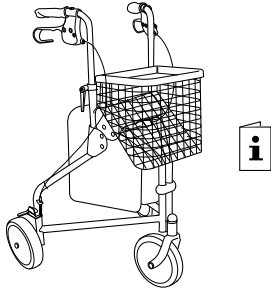- Rollator, partly assembled
- Basket, tray and bag
- User manual

I.5 Warranty

We supply a two year warranty from the date of delivery. Deviations may occur due to local agreements. Damage caused by normal wear to e.g. wheels, handles and brakes or damage caused by abnormal use (e.g. exceeded max. user weight) is not covered by the warranty. Maintenance and operating instructions must be followed for the warranty to be valid.

2 Safety

2.1 General warnings

WARNING!

Risk of falling

- Exercise caution when walking on sloping / slippery ground and on uneven surfaces.
- Be careful when walking rearwards over obstacles.
- Do not go up or down stairs with the rollator.
- Do not use the rollator on escalators.
- Do not under any circumstances stand on the rollator.
- Loads must only be transported in the basket, on the tray or in the bag. Be careful when there is a heavy load in the basket (for the max. load in the basket refer to chapter “Technical Data”).

WARNING!

Risk of injury

- Do not let children play with the rollator.
- If the push handles are exposed to extreme temperature (above 41°C) there might be risk for first degree burns.


IMPORTANT!

Risk of corrosion

- The rollator must not be stored outside in wet climate or in humid environments for a long period of time.

2.2 Labels and symbols on the product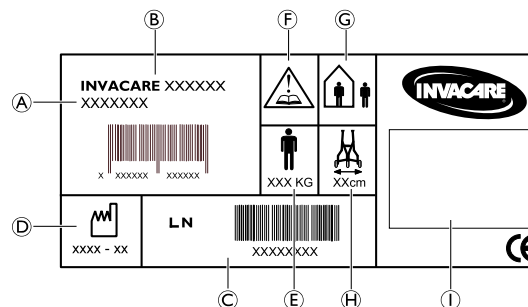

The label contains all important technical information regarding your walking aid.

- | | |
|-----------------------|-------------------------------|
| Ⓐ Article number | Ⓕ Refer to user manual |
| Ⓑ Product name | Ⓖ In- and outdoor walking aid |
| Ⓒ Serial number | Ⓗ Max. width of walking aid |
| Ⓓ Date of manufacture | Ⓘ Manufacturer's name. |
| Ⓔ Max. user weight | |

3 Usage

3.1 Before use

We do not recommend that the product is issued to anyone without first carrying out a full assessment of their specific needs. It is important that the user is given the necessary support by the product as prescribed by a qualified professional. This is both for the safety of the user and also their recovery pattern.

3.2 Operating position

WARNING!
Risk of falling


- Make sure that the safety latch is in a locked position before using the rollator.


Difficult to adjust lever

- The locking lever and washer must be mounted on the outside of the tube.

1.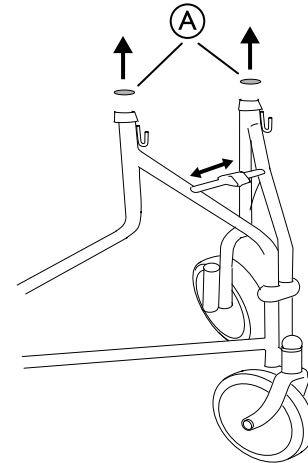

Remove the plastic caps **A** and open the rollator.

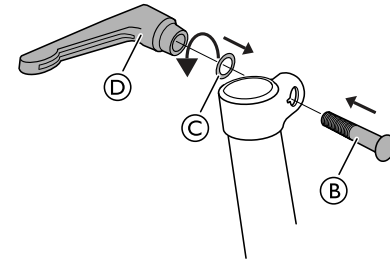
2. Attach the bolt **B** and the washer **C**.

Ensure that the bolt slots into the recess of the fitting.

3. Attach the locking lever ④ on the bolt ③.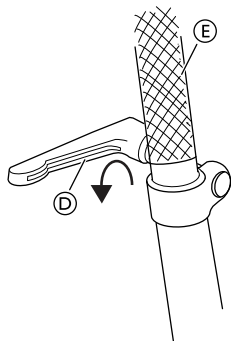

4. Fit the handle ⑤ to the tube.
5. Tighten the locking lever ④.


The locking levers must be pointing downwards during use.

6. Repeat the procedure on the opposite side.

3.3 Folding and unfolding the rollator

Folding the rollator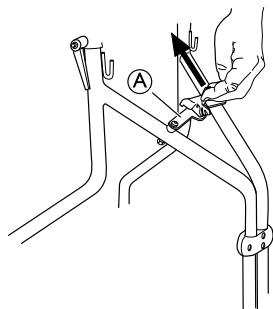

1. Pull the safety catch ① upwards - backwards.
2. Push the handles inwards to fold the rollator.

WARNING!

Risk of pinching

– Be aware of the risk of pinching body parts between the parts of the frame when folding the rollator.


Unfolding the rollator

WARNING!

Risk of falling


– Make sure that the safety latch is in a locked position before using the rollator.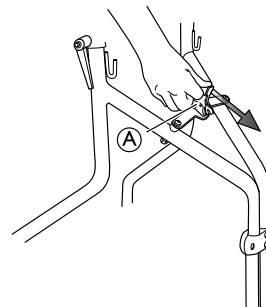

1. Push the handles outwards to unfold the rollator.
2. Pull the safety catch ① downwards - forwards to lock the rollator in an unfolded position.

3.4 Mounting the bag, basket and tray


Bag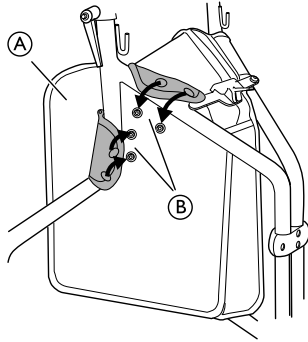

1. Mount the bag (A) between the frame tubes.
2. Fold the attachment flaps (B) over the tubes and secure the buttons.
3. Repeat the procedure on the opposite side.

Basket

1.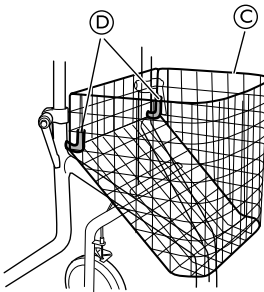


Attach the basket (C) on the attachment hooks (D).

1542458-C

Tray


1.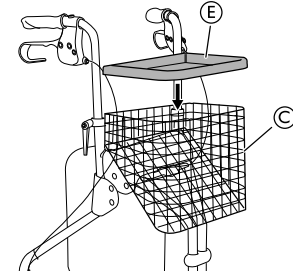


Place the tray (E) on the top of the basket (C).

3.5 Using the hand brakes

1.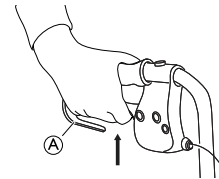

2.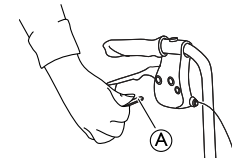

1. Apply the running brakes by pulling both brake handles (A) upwards.
2. To apply the parking brakes, press the brake handles (A) completely downwards.

An audible sound indicates that the brake is in the correct position.

 It is important that both brakes are put into parking position.


3. To release the parking brakes, pull the brake handles upwards.

Adjusting the brake effect

WARNING!
Risk of poor brake effect

– The adjustment of the brake must be done by qualified personnel.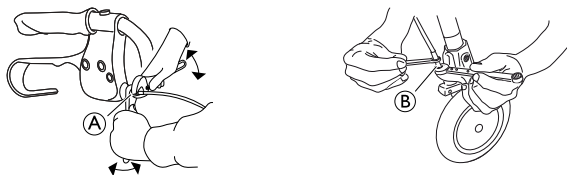


The brake can be adjusted both by the handle (A) or below by the wheel (B).

Spanner 8 mm and 10 mm (not included)

3.6 Adjusting the handle height/angle

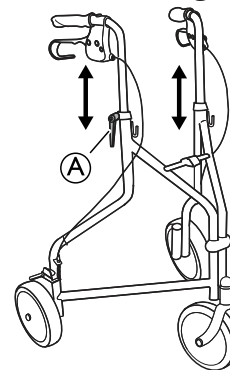
1. Stand behind the rollator with flat shoulders.
2. Loosen the lever (A).
3. Adjust the handle to suitable height and/or angle.

The handles should usually be positioned at wrist height. Therapist may suggest different heights depending on specific needs of the client.

4. Re-tighten the lever.
5. Repeat the procedure on the other side.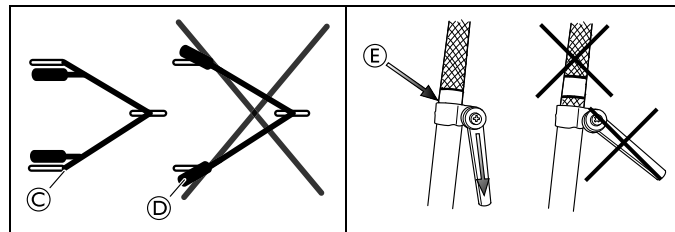


**WARNING!****Risk of falling**


- The handlebars must point directly backwards © when using the rollator. Under no circumstances should they be angled outwards ④.
- The handles must never be set above their highest position, indicated by the knurling on the tubes ⑤. Handles should be set to the same height on either side.

3.7 Walking**WARNING!****Risk of falling**

- Be careful if the wheels stop against an obstacle.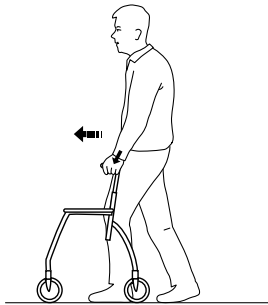


1. Place the walking aid a bit in front of you and take a step towards it whilst using the hand grips for support.

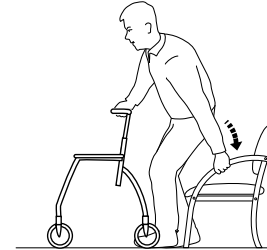3.8 Sitting down on a chair

1. Stand in front of the chair with the walking aid, the back of the legs should touch the chair.
2. Place the hands on the arms of the chair, lean forward slightly and sit down carefully.

3.9 Rising to a standing position**WARNING!****Risk of tipping**

- Never use the walking aid to pull yourself up from a sitting position.
- The parking brakes must be in braked position when the walking aid is used as a support when sitting down or getting up, from a chair for example.

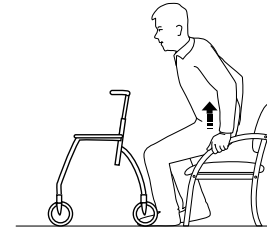1. Place the walking aid in front of you, use the armrests of the chair as support and rise carefully to a standing position.
2. Once steady in a standing position, transfer hands to the hand grips of the walking aid.

4 Maintenance and care

4.1 Maintenance

Maintenance should be done at least once a year.

To ensure good performance and safety you should regularly check that the following is carried out:

- Ensure that screws and adjustments are properly tightened.
- Never use a defective walking aid. Do not attempt repairs yourself!
- If hand grips are loose do not use the walking aid. Change hand grips according to the service instructions which can be obtained from your Invacare representative.
- Verify brake operation. If brake adjustment is required, see chapter 3 Usage, page 7 in this user manual.
- Periodically inspect the castors tightness, wear and damage.
- Make sure that the wheels are free of hair, lint and other debris.

4.2 Cleaning

The product can be cleaned and disinfected using commercially available agents.

IMPORTANT

- Cleaning must be done at least once a year.
- Do not use an abrasive detergent cleaner.
- Do not use a steam cleaner.
- Never use acids, alkalies or solvents.

- The product can be washed in a washing machine or by hand.
- The washing and drying temperature may not be higher than 60° C.

- High-pressure cleaner or ordinary jet of water may be used. However, do not aim the jet towards bearings, draining holes or labels.
- Clean with damp cloth and mild detergent.
- Clean the wheels with a brush (no wire brush!).
- After washing, wipe the product dry.

Disinfection

Disinfect the product by wiping down all generally accessible surfaces with disinfectant.

5 After Use

5.1 Disposal

When the product can no longer be used, it and its accessories as well as the packaging material should be sorted in the correct way.

The individual parts can be separated and handled according to their material.

Please be environmentally responsible and recycle this product through your recycling facility at its end of life.

5.2 Re-use

This product is suitable for re-use after the following has been performed:

- Inspection
- Cleaning

For details, see chapter 4 Maintenance and care, page 13.

WARNING!

Risk of injury


- Always inspect the product according to the service instructions before transferring the product to a different user.
- If any damage is found do not use the product.

The user manual belongs to the product. Make sure that the user manual will be handed over with the product.

6 Technical data

6.1 Dimensions and weight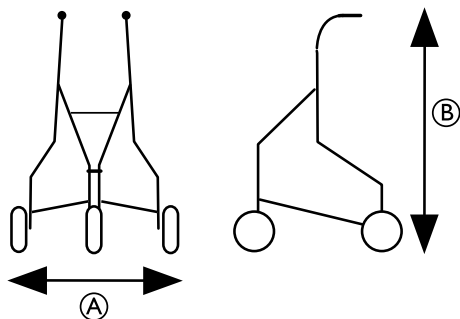

Delta P429/2 — P429/2C	
Total width Ⓐ	680 mm
Total height Ⓑ	830 — 950 mm
User Height Min. / Max	1600 — 1830 mm
Weight limitations	135 kg *
Max. load basket	5 kg
Max. load tray	5 kg
Max. load bag	5 kg

* The weight limitation includes the contents of the basket, tray or bag and the weight of the user.

6.2 Material

Frame	Polyester powder coated steel / Chrome plated steel
Handle bars	Chrome plated steel
Handgrips	PVC
Wheels	Polypropylene with treads of TPE
Plastic parts	Material according to marking (PA, PP, PE)
Screws	Steel, rust protected

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Invacare Poirier SAS
Invacare Austria GmbH
Invacare Rea AB