Invacare® Aqua / Ocea

H140, H141, H142 / H320/1

en	Grab handle User Manual	2
de	Haltegriff Gebrauchsanweisung	3
fr	Poignée de maintien Manuel d'utilisation	6
it	Maniglia di sicurezza Manuale d'uso	8
es	Asa de sujeción Manual del usuario	10
pt	Pega para agarrar Manual de utilização	12
nl	Handgreep Gebruiksaanwijzing	14

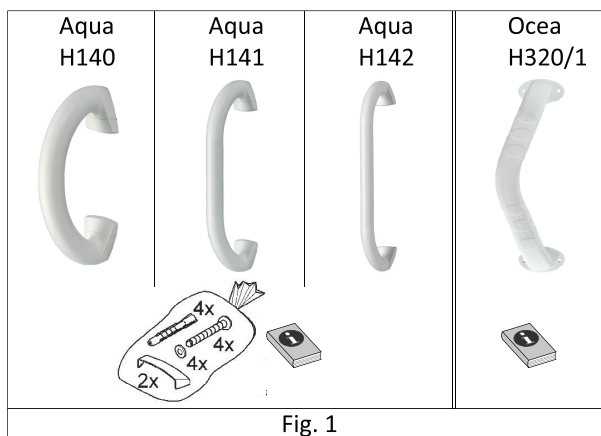
Fig. 1

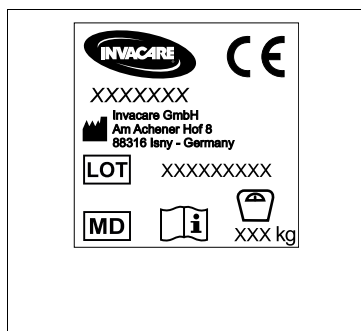
Fig. 2

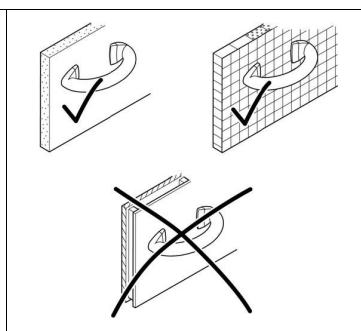
Fig. 3

Fig. 4

Fig. 5

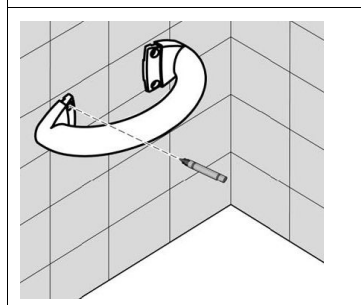
Fig. 6

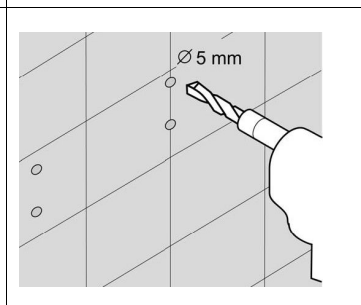
Fig. 7

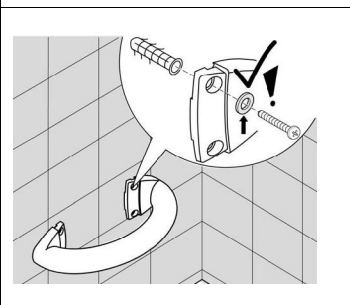
Fig. 8

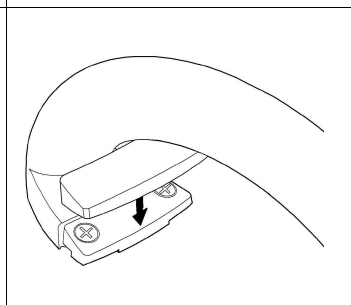
Fig. 9

This manual MUST be given to the user of the product.
BEFORE using this product, this manual MUST be read and saved for future reference.

Yes, you can.®

1 General

1.1 Introduction

This user manual contains important information about the handling of the product. To ensure safety when using the product, read the user manual carefully and follow the safety instructions. If you find that the font size in the printed document is difficult to read, you can download the PDF version from the website.

Invacare reserves the right to alter product specifications without further notice.

In case of a serious incident with the product, you should inform the manufacturer and the competent authority in your country.

1.2 Warranty Information

We provide a manufacturer’s warranty for the product in accordance with our General Terms and Conditions of Business in the respective countries. Warranty claims can only be made through the provider from whom the product was obtained.

1.3 Compliance

This CE-marked medical device conforms to: EU Medical Device Regulation, ISO 13485, ISO 14971, REACH, EN 12182 and related standards.

1.4 Service life

The expected service life of this product is five years when used daily in accordance with safety instructions and intended use, stated in this manual. The effective service life can vary according to frequency and intensity of use.

1.5 Limitation of Liability

Invacare accepts no liability for damage arising from:

- Non-compliance with the user manual
- Incorrect use
- Natural wear and tear
- Incorrect assembly or set-up
- Unauthorised modifications and/or use of unsuitable spare parts

2 Safety

2.1 Safety Information

WARNING!
Risk of Serious Injury or Damage

Improper use of this product may cause injury or damage.

- If you are unable to understand the warnings, cautions or instructions, contact a health care professional or provider before attempting to use this equipment.
- Do not use this product or any available optional equipment without first completely reading and understanding these instructions and any additional instructional material.

WARNING!
Risk of Falling!

- Before every use, check the product for damage and ensure that it is securely assembled.
- Do not use if defective.
- Do not make any unauthorised alterations or modifications to the product.
- Do not use as sports equipment or climbing aid.
- Only use suitable wall plugs and bolts for mounting the H320/1.

WARNING!
Risk of Falling!

- Excessive or intermittent loads can lead to breaking.
- Do not exceed the maximum load, refer to 7 *Technical Data, page 3*.

2.2 Labels and Symbols on Product

Identification Label (Fig. 2)

	Conformity mark		Maximum safe load
	Lot number		Read user manual
	Manufacturer		Medical Device

3 Components

3.1 Intended Use

Invacare Aqua (H140-H142) and Invacare Ocea (H320/1) are grab handles to be fixed on walls with screws. The products are intended as support aids for persons with mobility limitations when e.g. going in and out from a bathtub or moving in the shower.

Indication: If used without an assistant, the users must be able to keep their own balance. There are no contraindications known when using the product as intended. For the maximum user weight, see 7 *Technical Data, page 3*.

3.2 Scope of Delivery

The scope of delivery is shown in figure 1.

4 Setup

WARNING!
Risk of Falling!

- Mount only to walls that have a sufficient load-bearing capacity (Fig. 3).

1. Place the grab handle in the desired position against the wall, examples shown in Fig. 4 & 5, and mark the mounting points using a scribing tool (Fig. 6).
2. Use a drill to make suitably sized holes at the points marked on the wall (Fig. 7).
3. Fix the grab handle to the wall using the included wall mounting kit (Fig. 8).

A wall mounting kit is not included for model H320/1.

4. Only for H140 – H142: Place the screw covering caps on the fixing elements (Fig. 9).
5. Grip the grab handle with one hand and carry out the desired body movement (Fig. 4 & 5).

5 Maintenance

5.1 General Maintenance Information

This product is maintenance-free, provided that the cleaning and disinfection instructions are adhered to.

5.2 Cleaning and Disinfection

General Safety Information

**CAUTION!****Risk of Contamination**

- Take precautions for yourself and use appropriate protective equipment.

**NOTICE!**

Wrong fluids or methods can harm or damage the product.

- All cleaning agents and disinfectants used must be effective, compatible with one another and must protect the materials they are used to clean.
- Never use corrosive fluids (alkalines, acid etc.) or abrasive cleaning agents. We recommend an ordinary household cleaning agent such as dishwashing liquid.
- Never use a solvent (cellulose thinner, acetone etc.) that changes the structure of the plastic or dissolves the attached labels.

For cleaning and disinfection in clinical or long-term care environments, follow your in-house procedures.

Cleaning Intervals

**NOTICE!**

Regular cleaning and disinfection enhances smooth operation, increases the service life and prevents contamination.

Clean and disinfect the product:

- before first use,
- regularly while it is in use,
- when it has been in contact with any body fluids.

Cleaning

**NOTICE!**

- Maximum cleaning temperature:
 - 60 °C by hand

1. Clean the product by wiping down all generally accessible surfaces.
2. Dry the parts with a cloth.

Disinfection Instructions

Method: Follow the application notes for the used disinfectant and wipe-disinfect all accessible surfaces.

Disinfectant: Ordinary household disinfectant.

Drying: Allow the product to air-dry.

6 After Use

6.1 Reconditioning

**WARNING!****Risk of Falling**

Worn fastening elements can suddenly come loose and cause falling.

- Always use new wall plugs and bolts when repositioning the product.

This product is suitable for reuse. To recondition the product for a new user, carry out the following actions:

- Inspection
- Cleaning and disinfection

For detailed information, see *5 Maintenance, page 3*.

Make sure that the user manual is handed over with the product.

6.2 Disposal

Disassemble the product and its components, so the different materials can be separated and recycled individually.

The disposal and recycling of used products and packaging must comply with the laws and regulations for waste handling in each country. Contact your local waste management company for information.

7 Technical Data

7.1 Dimensions, Weights and Materials

Product	Length [mm]	Max. load force	Product weight [kg]	Materials*
H140	250	120 kg	0.3	ABS
H141	400		0.5	
H142	500		0.6	
H320/1	330 (angled 135°)	115 kg	0.18	PP

The tube diameter of H140 – H142 is 35 mm, the oval shape of H320/1 is 40 mm x 28 mm

*All product components are corrosion resistant and do not contain natural rubber latex.

7.2 Environmental Parameters

Operating Conditions	
Ambient temperature	10 - 40 °C
Relative humidity	15% - 93%, non-condensing
Storage and Shipping Conditions	
Ambient temperature	0 - 40 °C
Relative humidity	30% - 75%, non-condensing

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Invacare Austria GmbH
Invacare Poirier SAS
Invacare GmbH