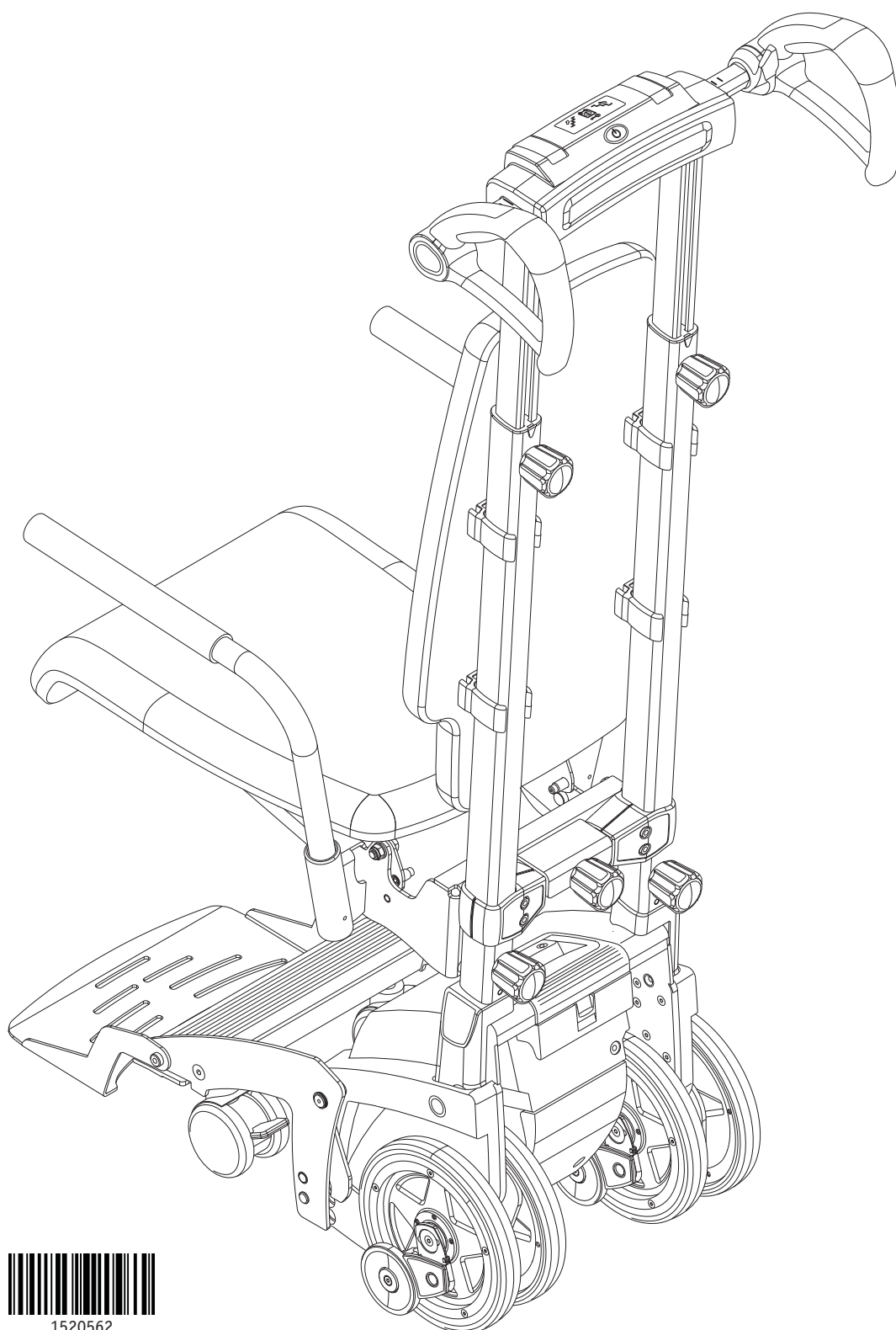
scalacombi S36/S39

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



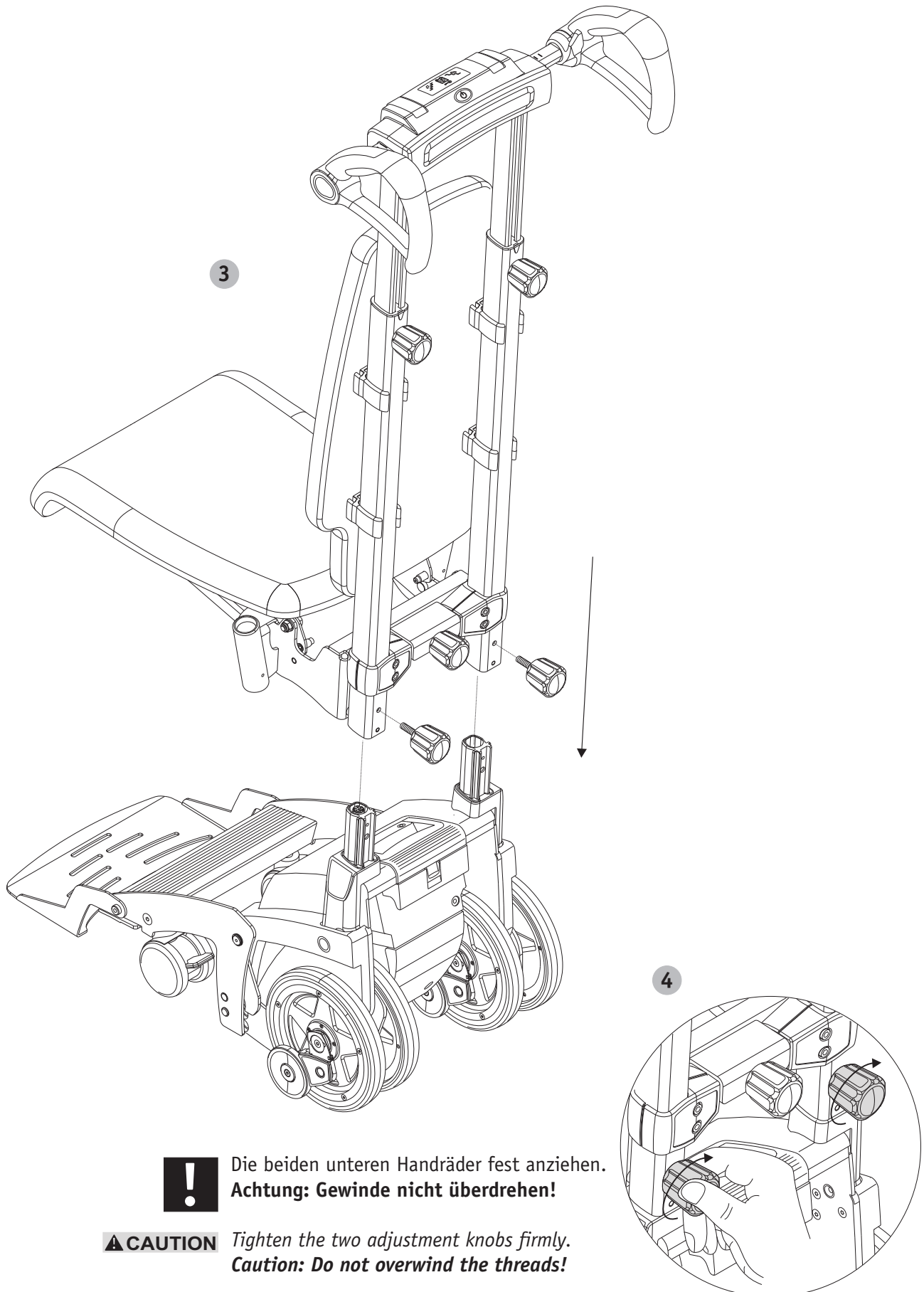
CAUTION *Tighten the adjustment knob firmly.
 Caution: Do not overwind the thread!*

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

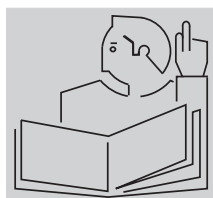
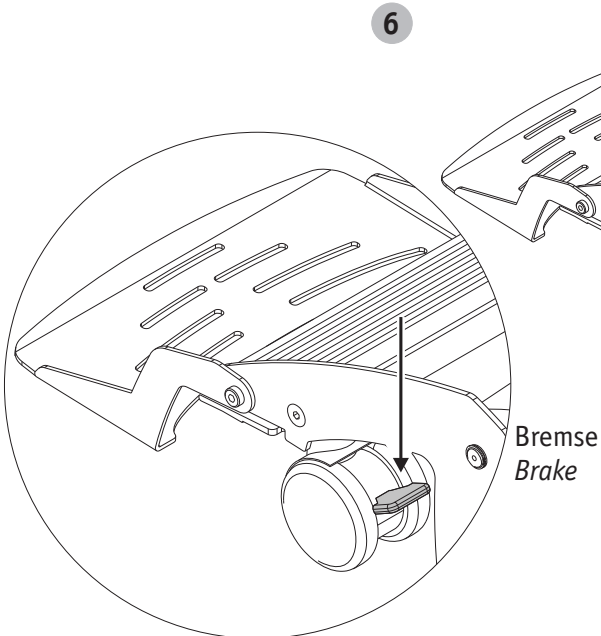
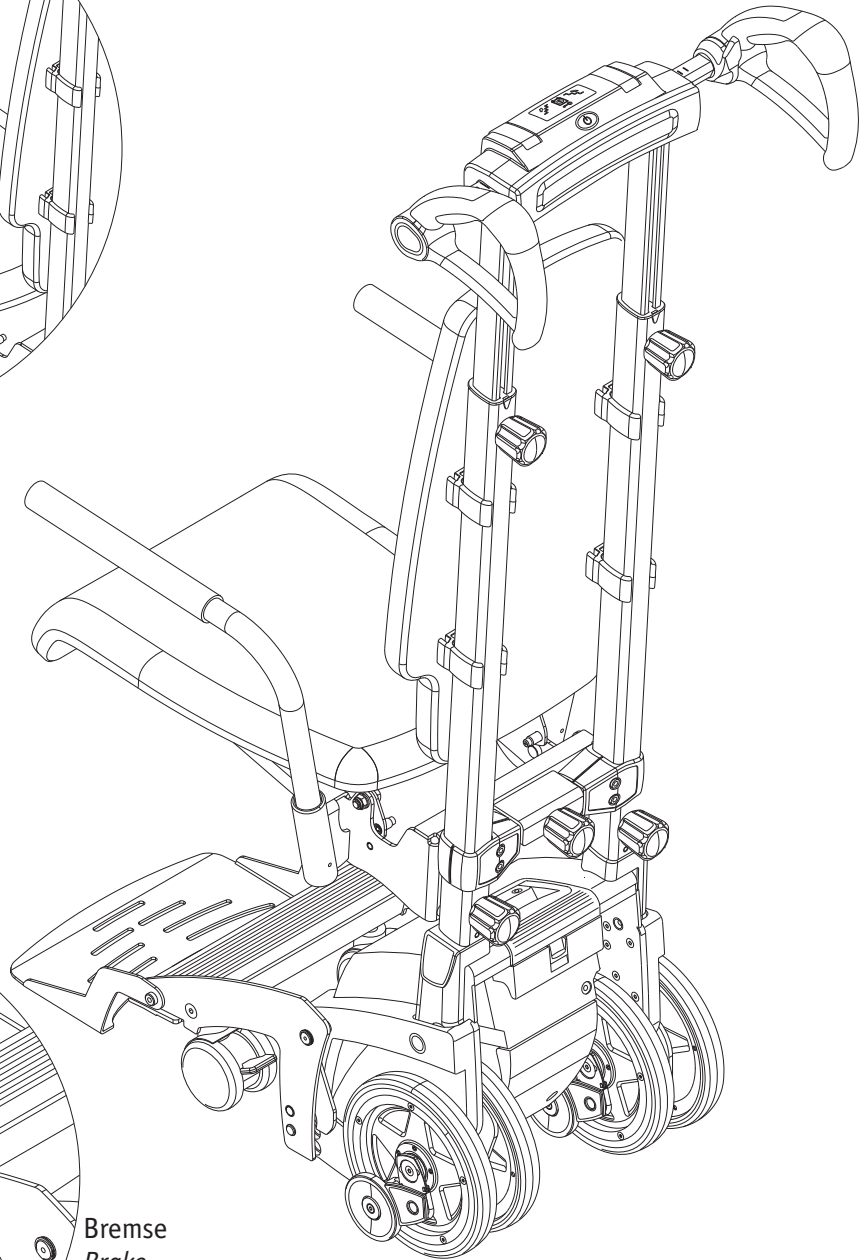
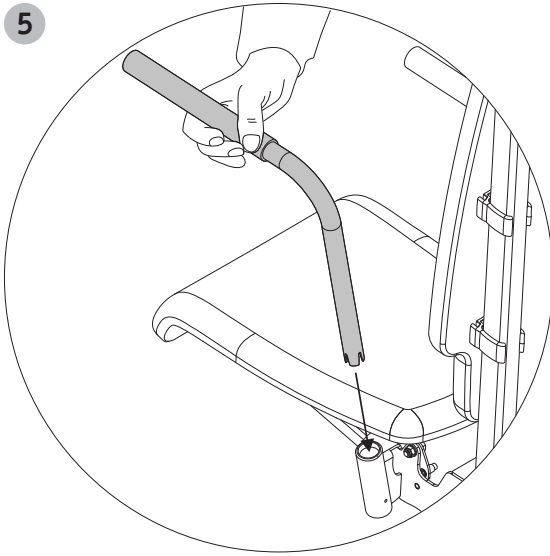
CAUTION Tighten the two adjustment knobs firmly.
Caution: Do not overwind the threads!

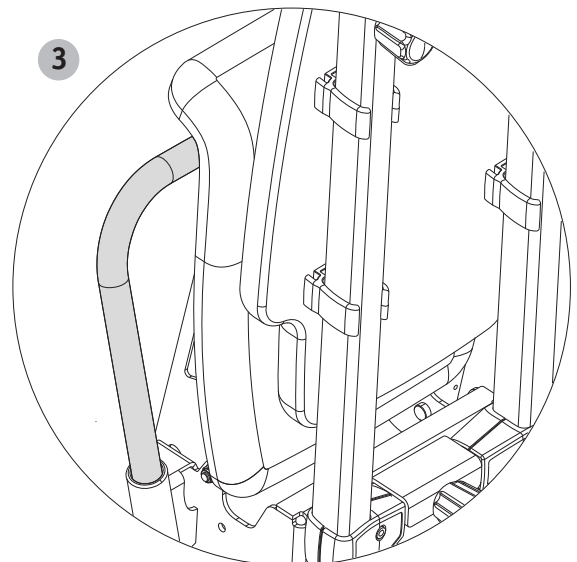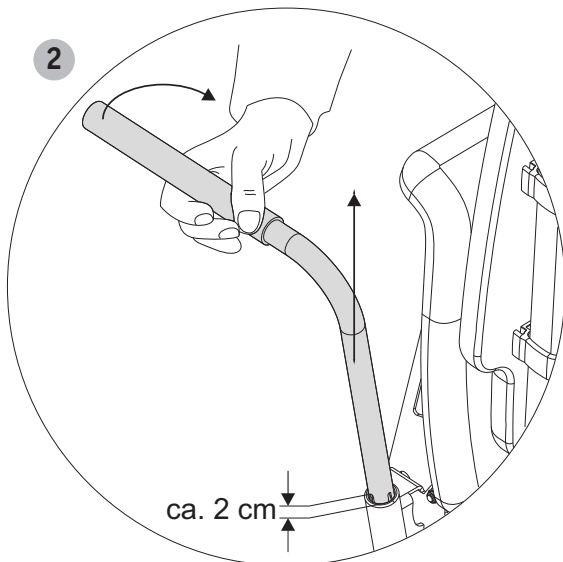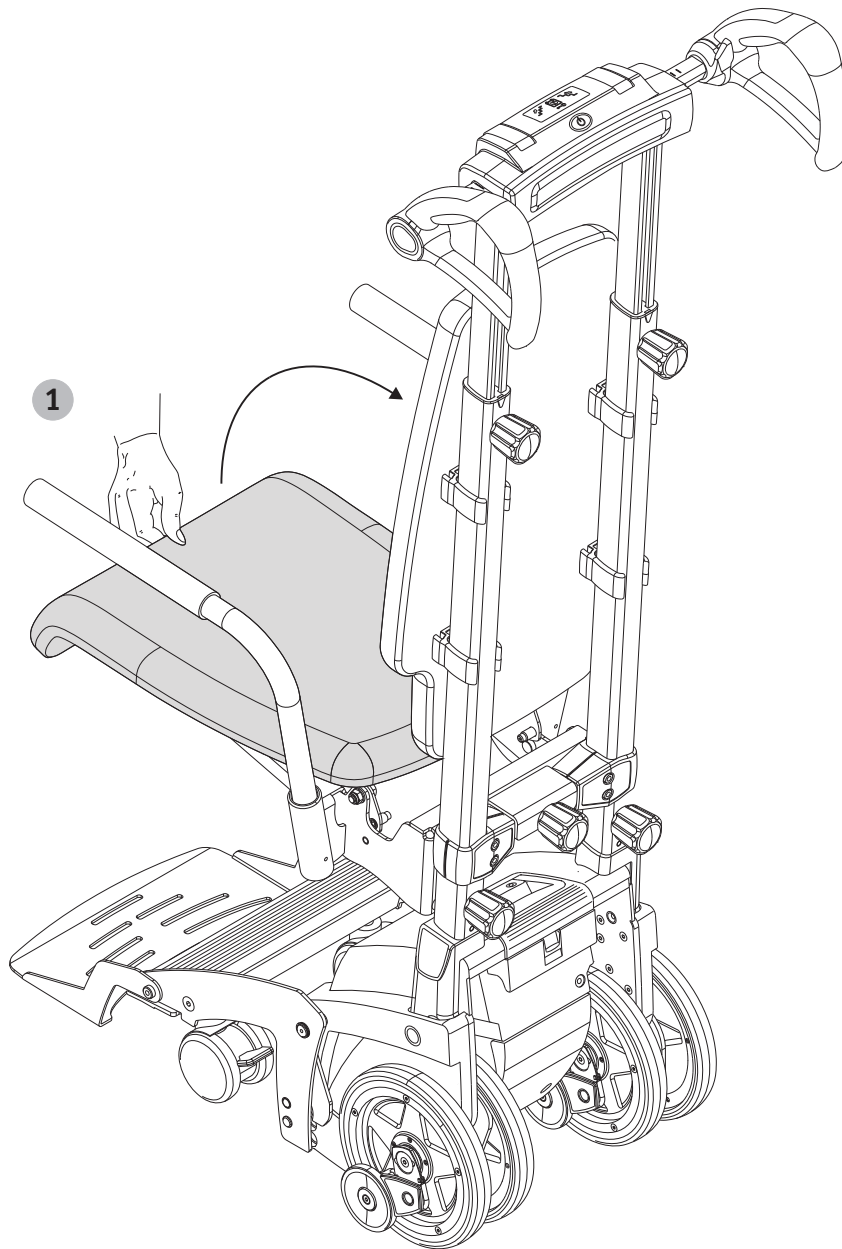scalacombi S36/S39 gebrauchsfertig; Gebrauchsanweisung lesen!
scalacombi S36/S39 ready to be used; read the user manual

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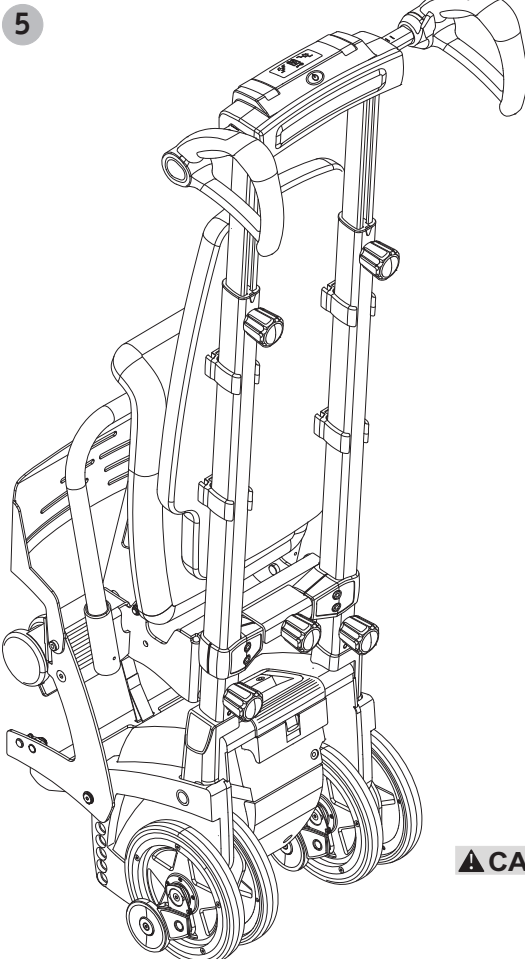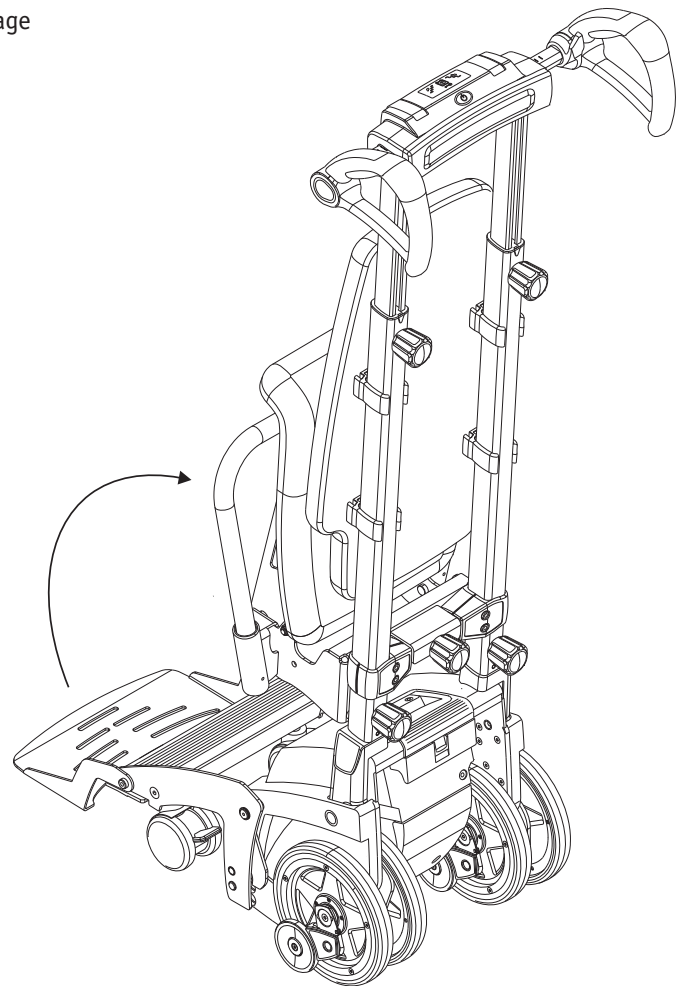
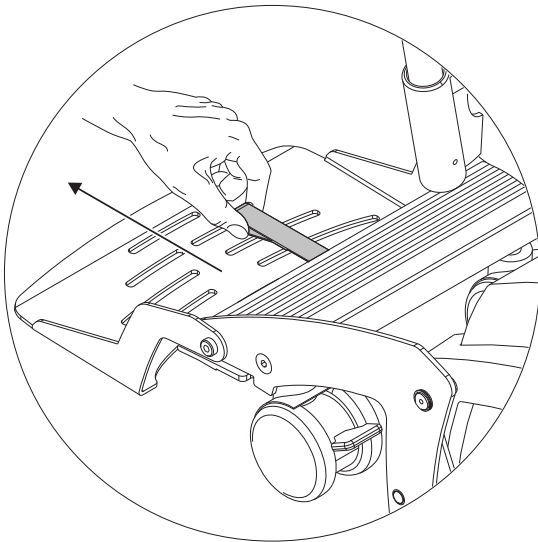
CAUTION

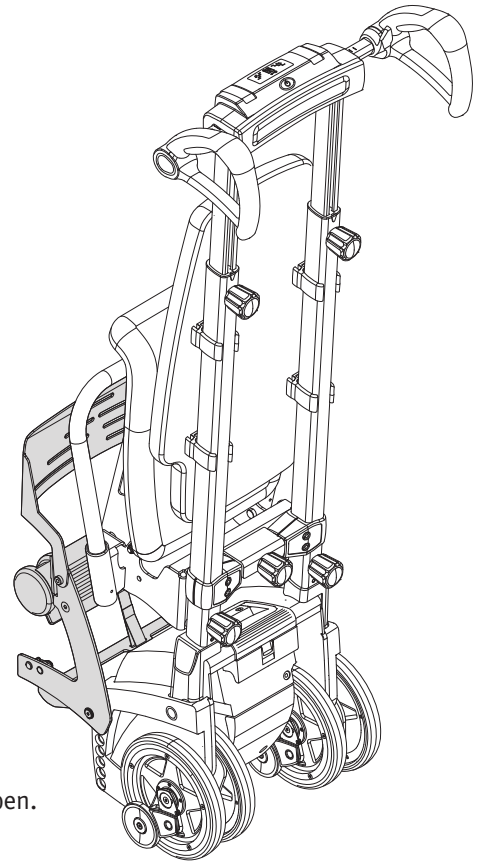
*Check the correct locking of the footrest.
It may not be loose and turn downwards.*

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


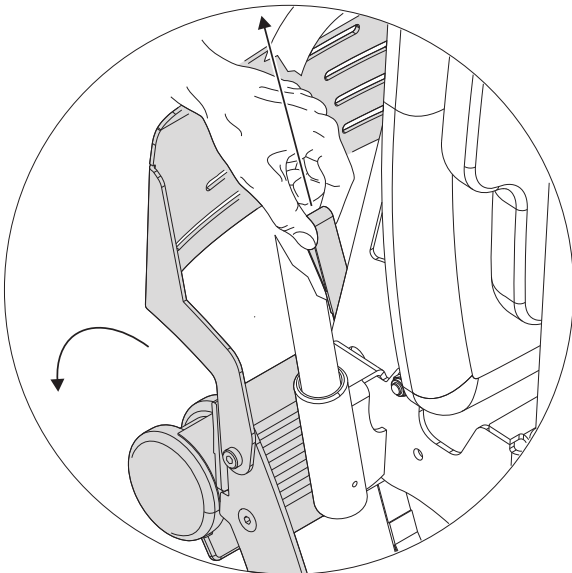
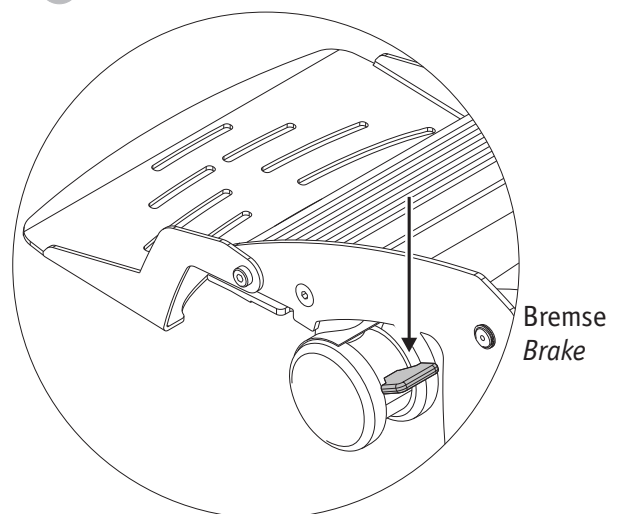
CAUTION

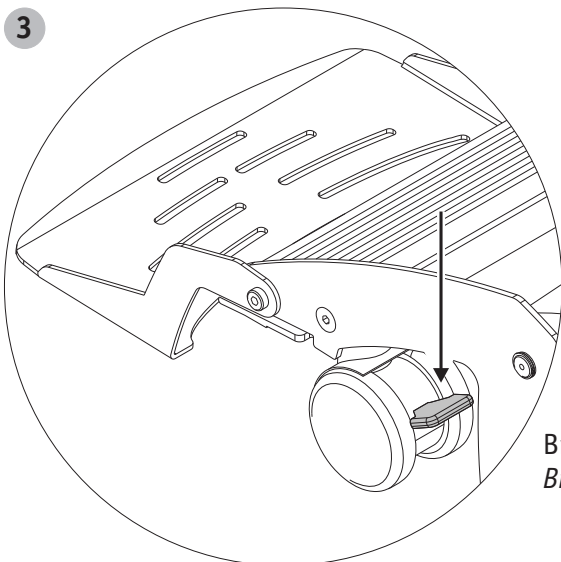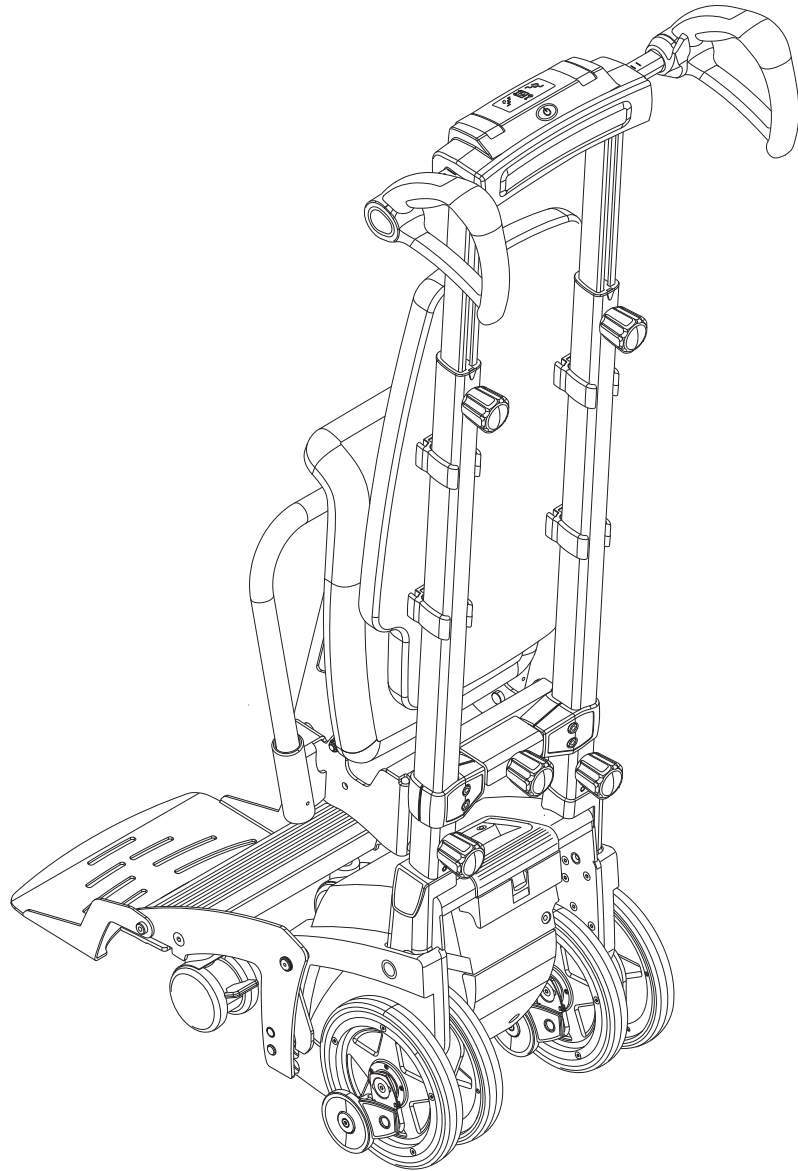*Check the correct locking of the footrest.
 It must not self-loosening and turn downwards.*

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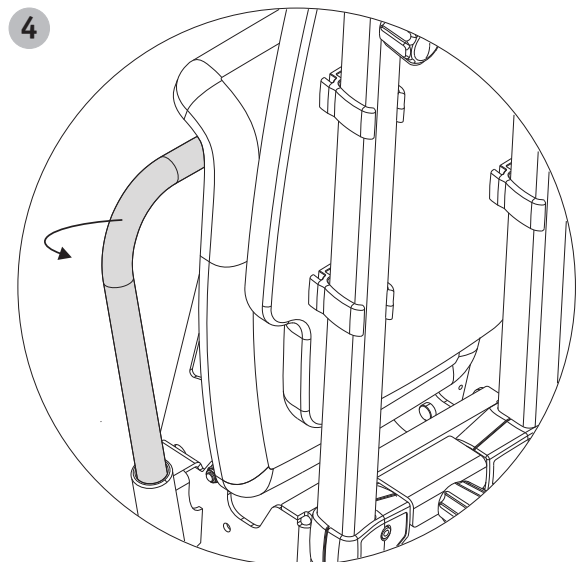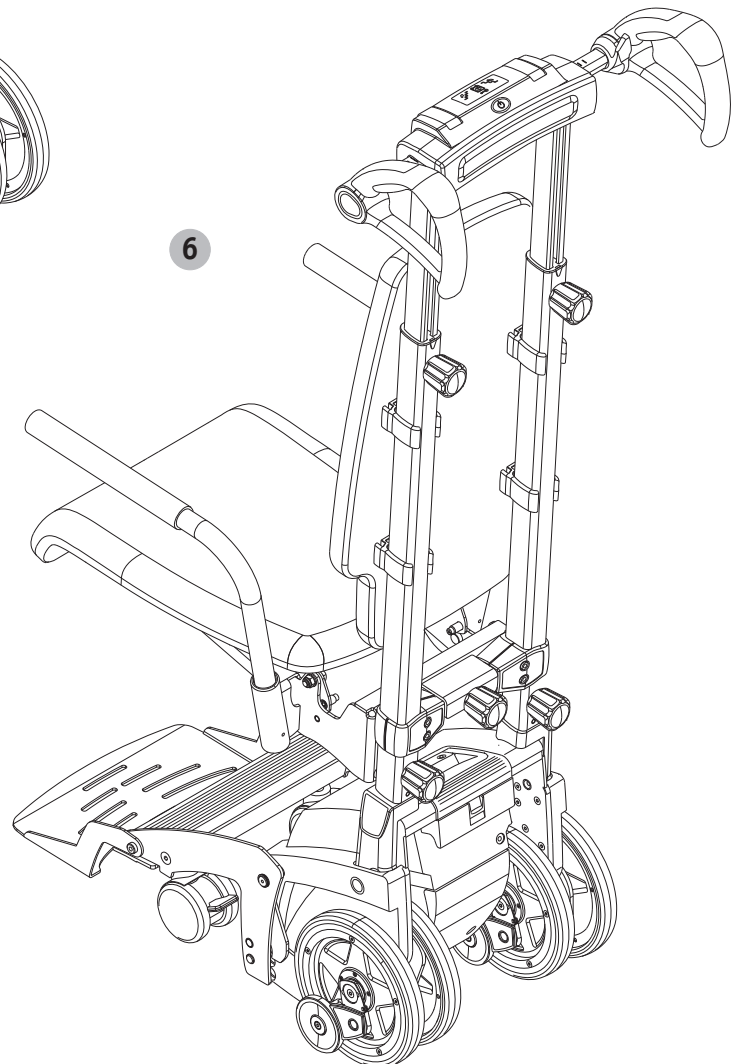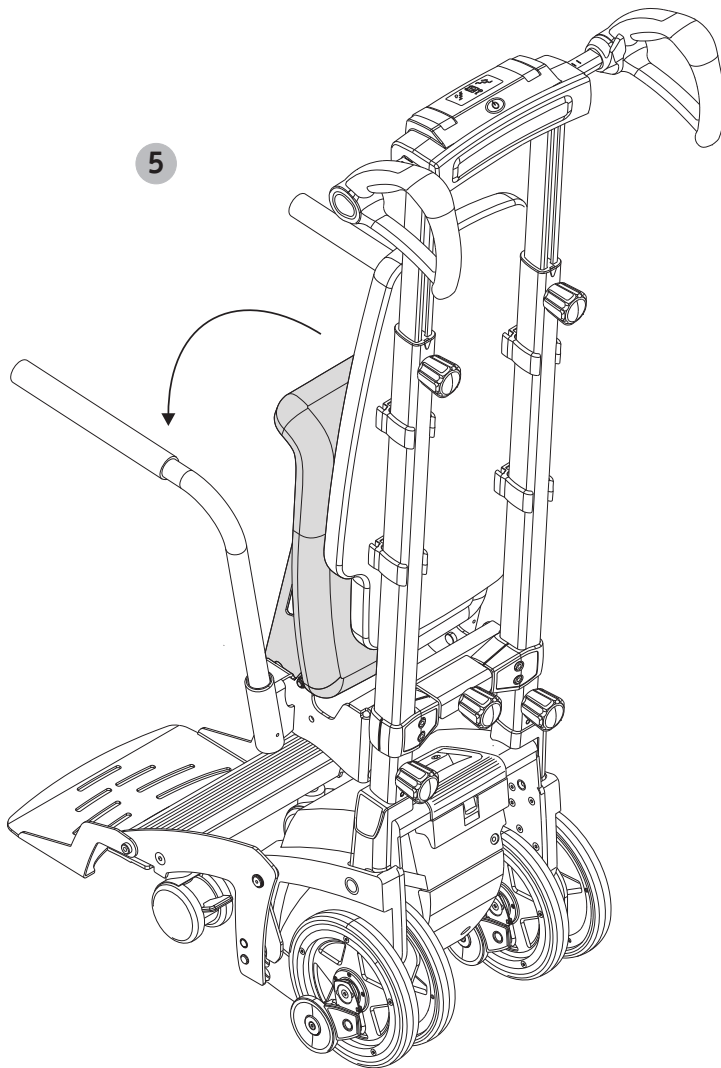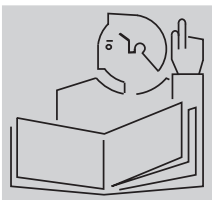
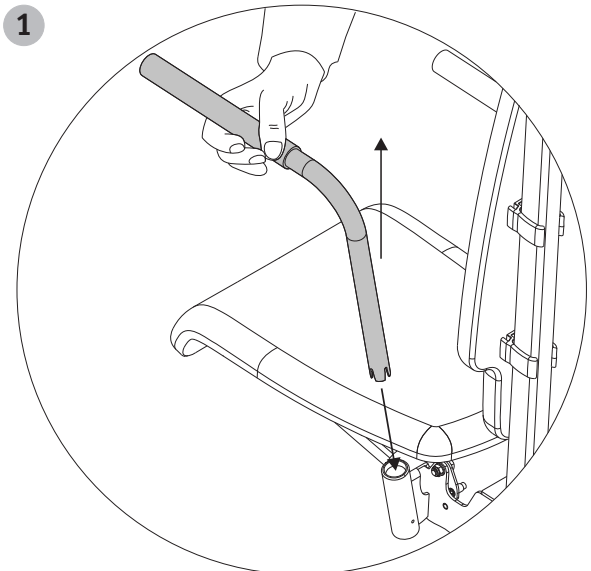
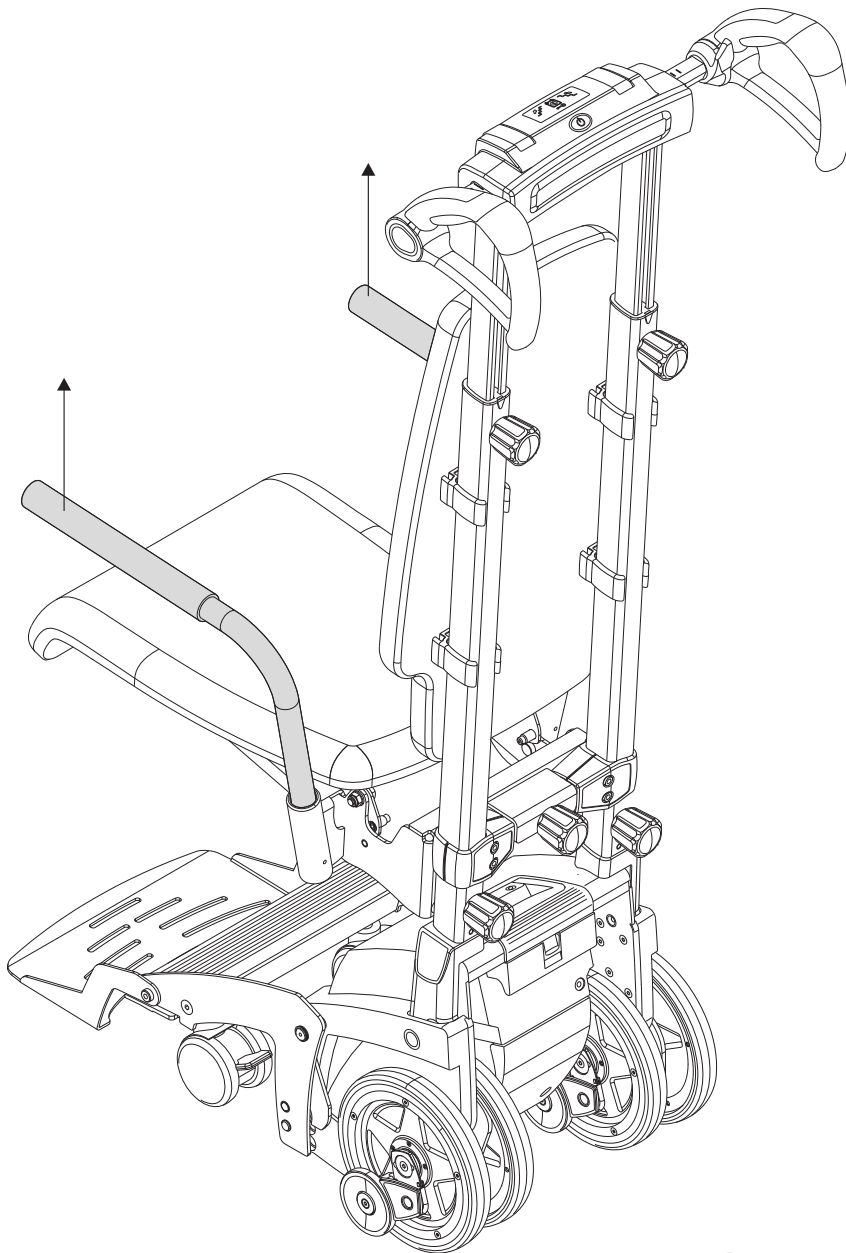
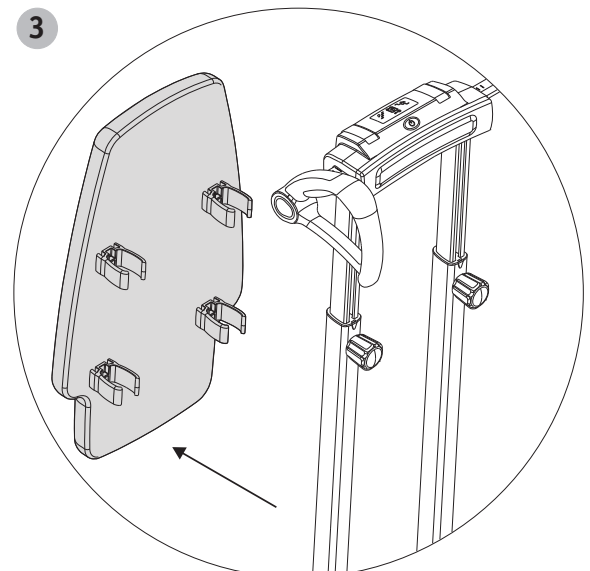
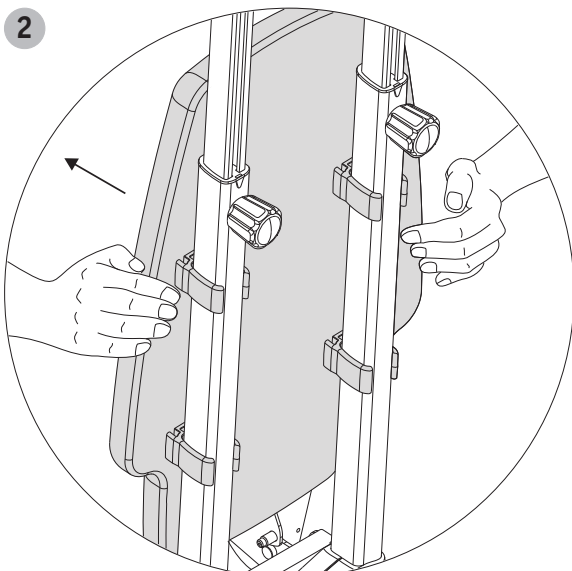
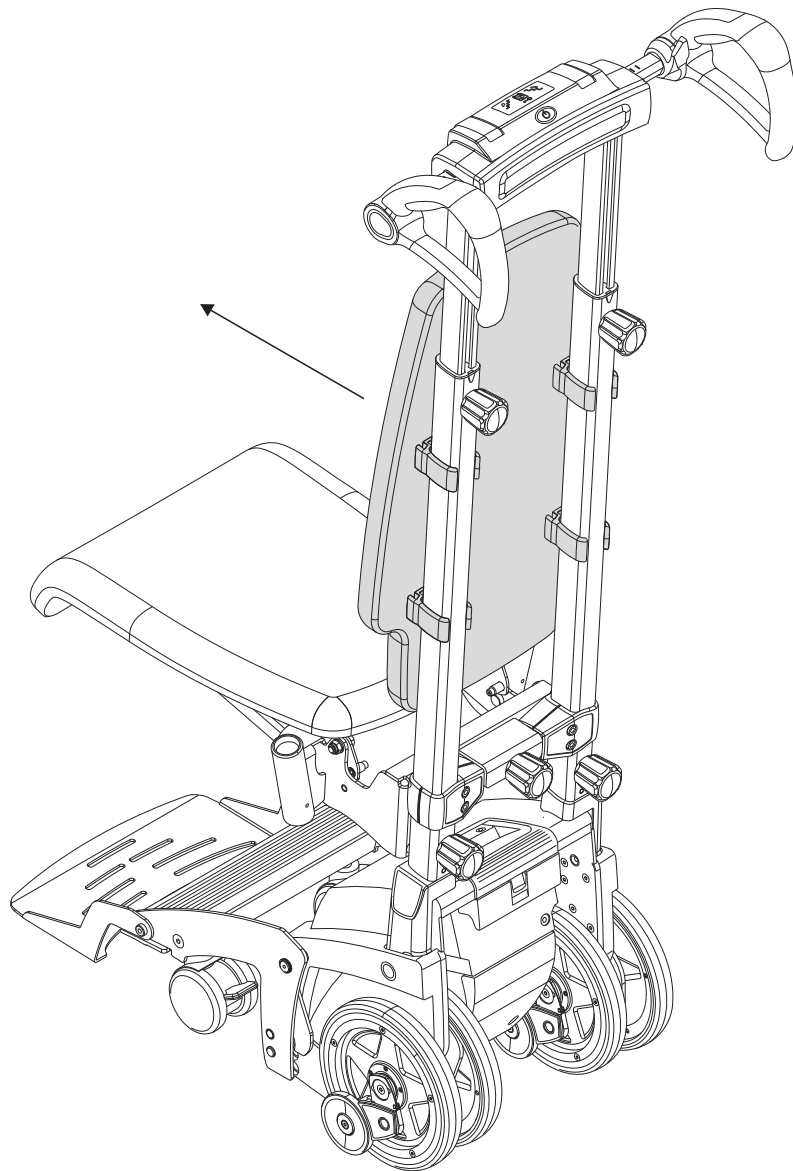
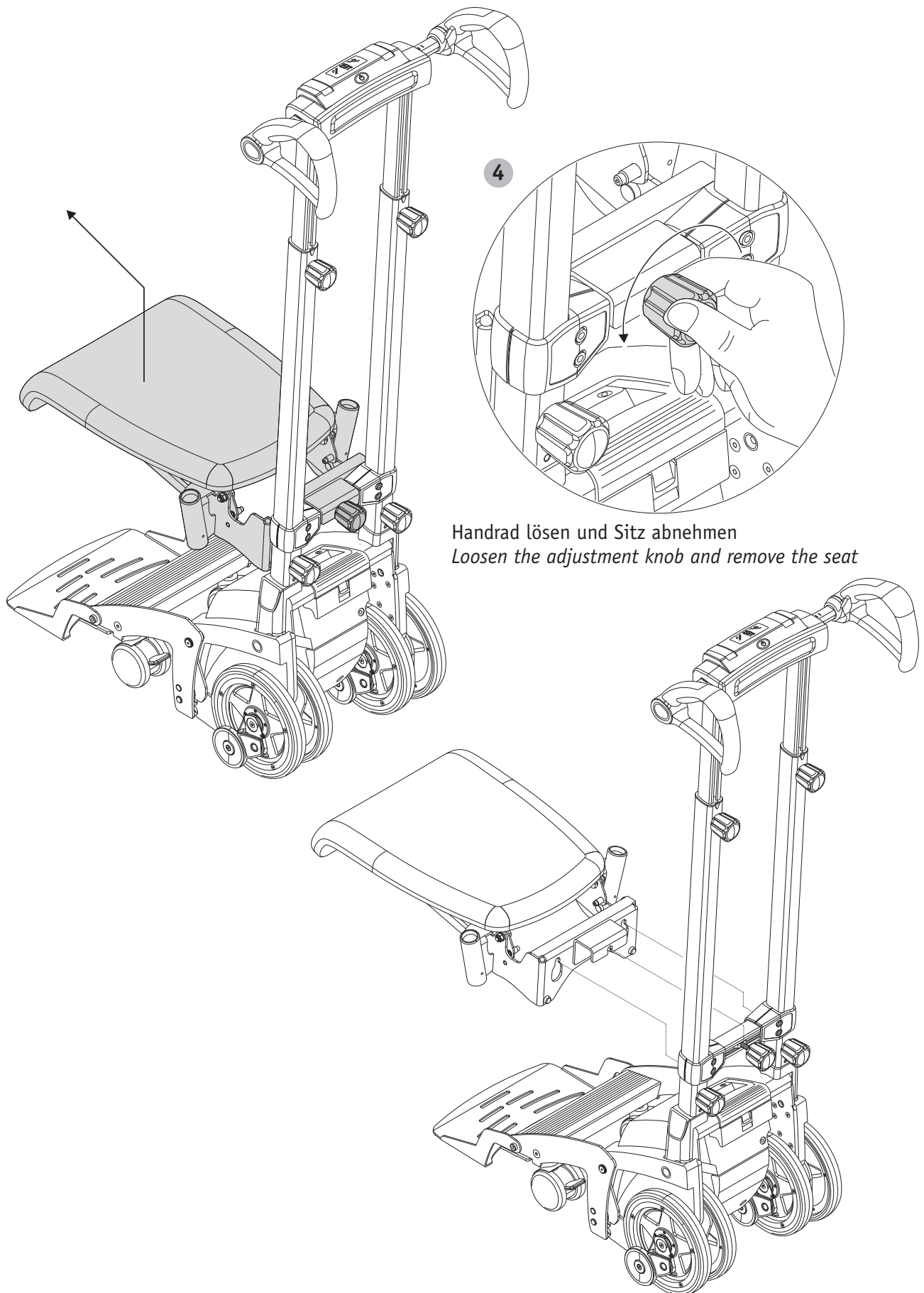
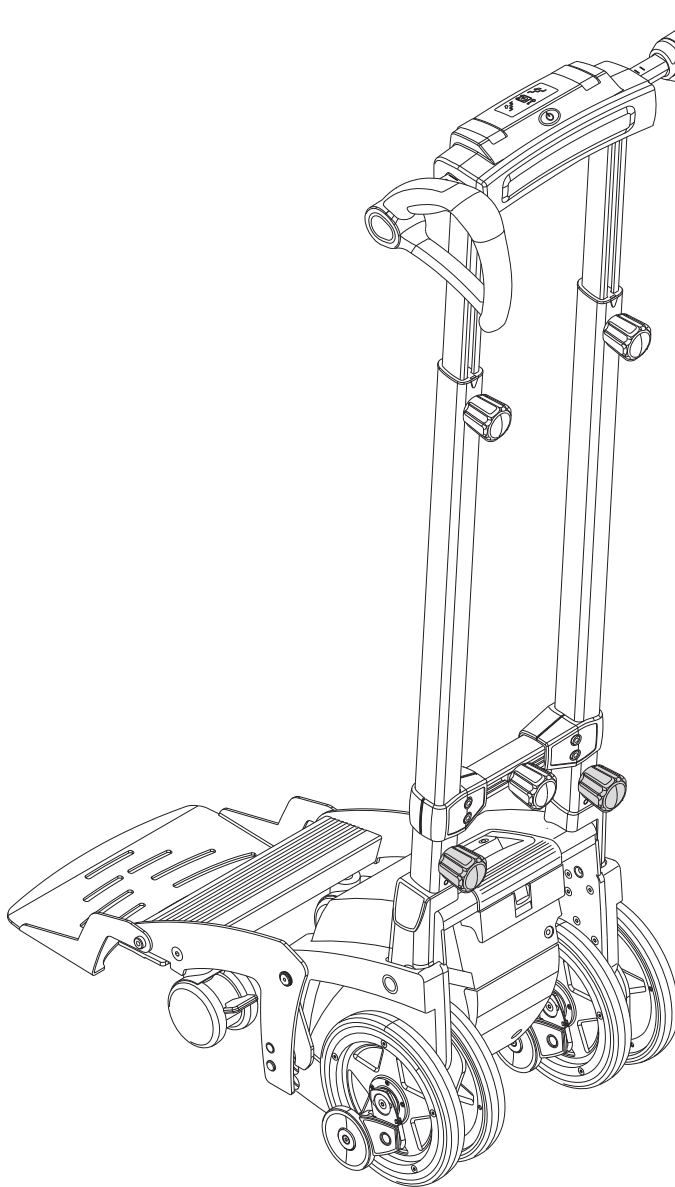
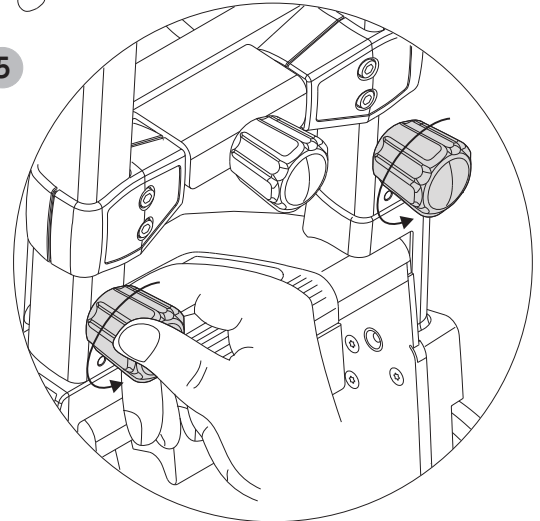
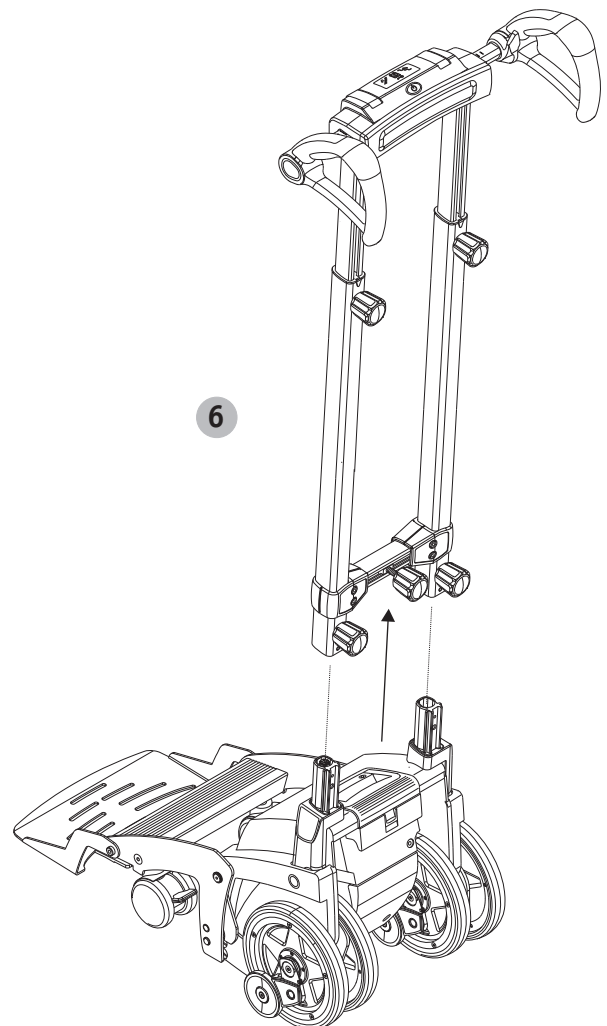
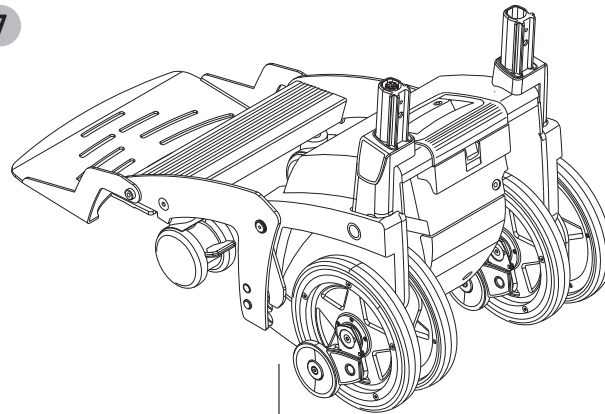
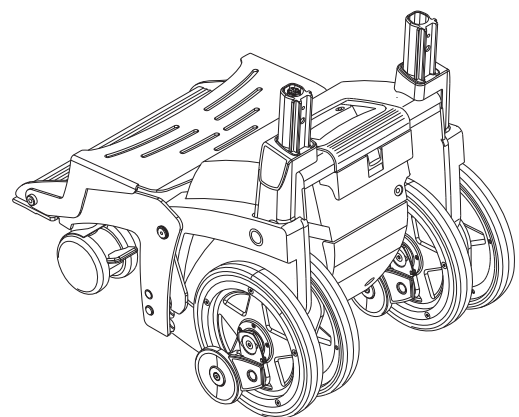
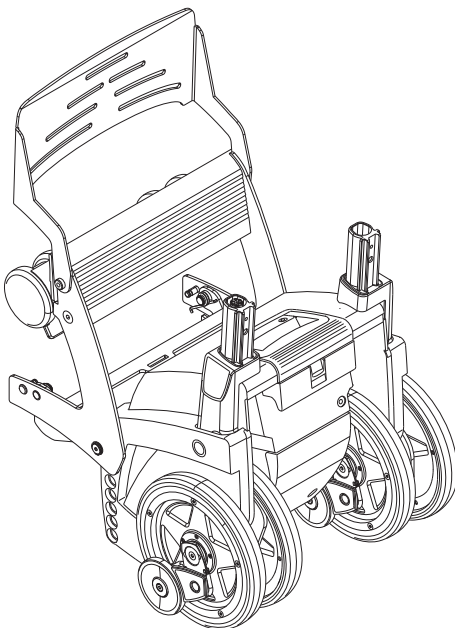
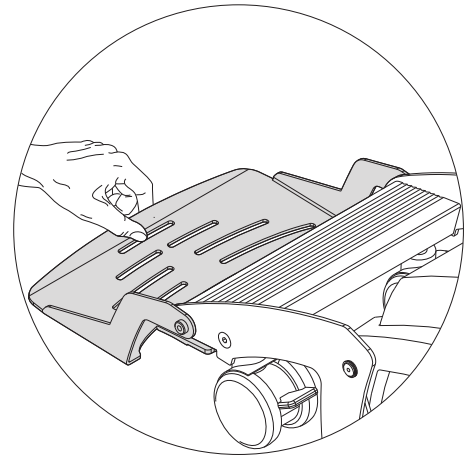
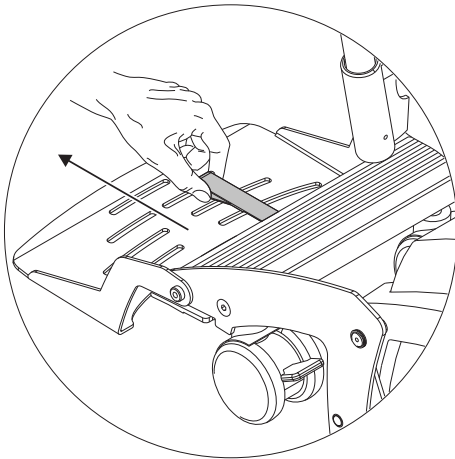
WARNING

- The two armrests attached to scalacombi serve to confine the seating area. They may also be used to rest the occupant's arms on.
- The max. load per armrest is limited to 50 kg.
- Do not load the armrest partial (tilting danger).
- Please do not sit on armrests.
- Please do not hang or put any objects (like e.g. bags) on the armrests.
- Please make sure that the armrests are locked correctly in the designated collet. They should not turn.
- Please do not hang or put any objects (like e.g. bags) on the armrests.
- While scalacombi is in use the arms of the occupant may not hang over the armrests.
- While scalacombi is in use the armrests may neither be removed nor be pivoted.
- Please read the user manual of scalamobil carefully before initial operation of scalacombi, since this document contains important information how to use the stair climbing device and provide additional safety advices.