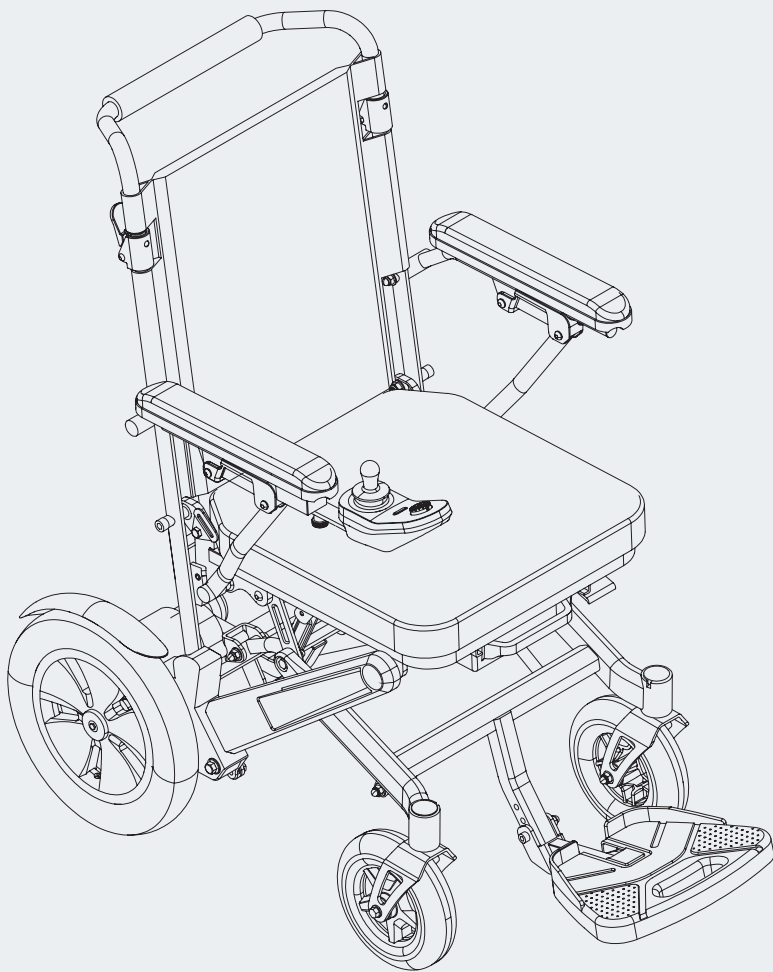
alber®

erivo®

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Ireland:

Invacare Ireland Ltd
Unit 5 Seatown Business
Campus
Seatown Road, Swords,
County Dublin
Tel: (353) 1 810 7084
Fax: (353) 1 810 7085
ireland@invacare.com
www.invacare.ie

Österreich:

Invacare Austria GmbH
Herzog-Odilo-Straße 101
A-5310 Mondsee-Tiefgraben
Tel: (43) 6232 5535 0
Fax: (43) 6232 5535 4
info-austria@invacare.com
www.invacare.at

Schweiz / Suisse / Svizzera:

Invacare AG
Benkenstrasse 260
CH-4108 Witterswil
Tel: (41) (0)61 487 70 80
Fax: (41) (0)61 487 70 81
switzerland@invacare.com
www.invacare.ch

Eastern Europe, Middle East & CIS:

Invacare EU Export
Am Achener Hof 8
D-88316 Isny
Tel: (49) (0)7562 700 397
eu-export@invacare.com
www.invacare-eu-export.com

Italia:

Invacare Mecc San s.r.l.
Via dei Pini 62,
I-36016 Thiene (VI)
Tel: (39) 0445 38 00 59
Fax: (39) 0445 38 00 34
italia@invacare.com
www.invacare.it

Portugal:

Invacare Lda
Rua Estrada Velha, 949
P-4465-784 Leça do Balio
Tel: (351) (0)225 193 360
Fax: (351) (0)225 1057 39
portugal@invacare.com
www.invacare.pt

United Kingdom:

Invacare Limited
Pencoed Technology Park,
Pencoed
Bridgend CF35 5AQ
Tel: (44) (0) 1656 776 222
Fax: (44) (0) 1656 776 220
uk@invacare.com
www.invacare.co.uk

EC	REP
----	-----

European representative:

MedNet EC-REP GmbH
Borkstrasse 10
48163 Muenster
Germany

CH	REP
----	-----

Representative Switzerland:

MedNet SWISS GmbH
Bäderstrasse 18
5400 Baden, Aargau
Switzerland

UK	RP
----	----

Responsible Person United Kingdom:

Medimap Ltd
2 The Drift
Suffolk
Thurston IP31 3RT
United Kingdom

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
Contents

1. Introduction	2	6. Warnings and error messages	20
1.1. Intended purpose	2	6.1. Error display on the On/Off button [18]	20
1.2. Important safety notes – please observe at all times	2	6.2. Error display by the LEDs [19]	20
1.3. Proper use of the Erivo	2	7. Automatic shutdown	21
1.4. Handling instructions	2	7.1. Shutdown due to internal error	21
1.5. Signs and symbols	3	7.2. Shutdown if not in use	21
1.6. Permissible conditions of use/locations of operation	4	8. Air pressure in the tyres	21
1.7. Standard scope of delivery	5	9. Adjusting the seating position	21
1.8. Optional accessories	5	10. Removing the seat cushion	21
1.9. The main elements at a glance	5	11. Transport	22
2. Commissioning	6	11.1. Folding the wheelchair	22
2.1. Switching on the system	6	11.2. Unfolding the wheelchair	22
2.2. Range	7	11.3. Safety instructions for loading the wheelchair	23
2.3. Switching off the system	7	12. Storage, care, re-use, maintenance and disposal	24
2.4. Travelling in push mode	7	12.1. Storage	24
3. Safety instructions and hazard information when using the Erivo	8	12.2. Care	24
3.1. General information	8	12.3. Reuse	25
3.2. Information about user training	8	12.4. Maintenance	25
3.3. Safety instructions	9	12.5. Disposal	25
3.4. Special safety instructions for travelling uphill, downhill and on ramps	10	13. Product safety information	25
3.5. Obstacles	10	14. Product service life	25
3.6. Hazardous areas and hazardous situations	10	15. Warranty, guarantee and liability	26
3.7. Electromagnetic interference (EMI)	12	15.1. Warranty for defects	26
4. Battery pack	13	15.2. Durability warranty	26
4.1. Removing the battery pack	13	15.3. Liability	26
4.2. Installing the battery pack	13	16. Important legal information for the user of this product	26
4.3. Charging the battery pack	13	17. Technical data	27
4.4. Capacity display	15	18. Labels	28
4.5. General information about charging the battery pack	15		
4.6. Safety instructions for the charger and the charging process	15	Appendix	
4.7. Safety instructions for the battery pack	16	A - Checks and inspections	30
4.8. Storing the battery pack	17	B - INVACARE representatives and dealers	32
4.9. Safety instructions and precautions on how to transport and ship the battery pack	17		
4.10. Important information about air transport	17		
5. Control unit	18		
5.1. Switching on the system	18		
5.2. Switching off the system	18		
5.3. Using the joystick to travel	18		
5.4. Warning signal	19		
5.5. Limiting the speed	19		
5.6. Swivelling the control unit	19		

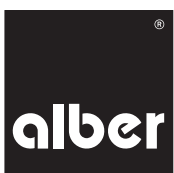1. Introduction

1.1. Intended purpose

The Erivo is an electric wheelchair. It is intended as a medical aid for disabled persons and serves to enhance their mobility and flexibility.

1.2. Important safety notes – please observe at all times

For safety reasons, the Erivo must only be operated by persons who

- have been instructed in its handling or / and have at least read and understood the entire operating instructions,
- can move and coordinate both hands or arms without major restrictions,
- have the physical and mental ability to safely operate the wheelchair in all possible situations (e.g. road traffic) and, in the event of the electrically driven wheels failing to work, are able to brake the wheelchair and stop safely,
- are not under the influence of medication (which impairs reaction times) or alcohol.

If you feel a little unsure about how to handle your Erivo, please contact your specialist dealer for advice.

When operating the Erivo, the specifications (e.g., maximum slope, air pressure of front wheels, etc.), as well as the general operating instructions, must be strictly adhered to. Never exceed any of the specified limit values.

We recommend refraining from trips in locations that are exposed to strong electric fields.

In rare cases, the operation of the Erivo may interfere with other electrical devices such as anti-shoplifting barriers in department stores. You must not use the Erivo on escalators or moving walkways.

If carrying out sporting activities in the wheelchair, such as lifting weights or similar, switch off the Erivo.

Similarly, combining the Erivo with accessories that have not been approved by Alber is also prohibited.

2

Commissioning the Erivo or operating it without reading and understanding the operating instructions is not permitted and constitutes improper use.

Any use contrary to proper use involves a risk of injury.

1.3. Proper use of the Erivo

The Erivo is designed exclusively for transporting persons with a walking disability. The Erivo must be used, transported, maintained and serviced as described in this operating manual.

1.4. Handling instructions

Do not attach any parts other than the accessories that have been approved for use by Alber. Similarly, the Erivo and its accessories must not be technically modified.

The Erivo must only be operated under the following conditions:

- Full compliance with the information, instructions and recommendations contained in this operating manual
- The Erivo is only to be handled by an instructed person or a person who has read and understood the entire operating instructions and is thus qualified to use it.
- No technical modifications have been made to the Erivo by the user or by any third party

An instructed person is a person who has been demonstrably instructed in the operation of the Erivo and in their tasks and who has been made aware of the possible hazards associated with improper use of the device or who has qualified himself/herself to use the Erivo by reading and understanding the entire operating instructions.

It is strictly prohibited for any uninstructed or unqualified persons to use the Erivo.

The Erivo must not be used for any purpose that contravenes the intended use of the device. This particularly applies to all types of load transportation such as transporting household objects or additional persons (e.g. children). Proper use includes complying with the information specified in this operating manual on carrying out maintenance work and observing and adhering to the safety precautions and hazard information for travel mode.

The use of the Erivo for any of the following purposes shall be deemed to be misuse by Alber GmbH:

- Use of the device that contravenes the instructions and recommendations in this operating manual
- Exceeding the technical limits laid down in this operating manual
- Technical modifications to the device
- Attachment and/or use of parts and accessories that have not been supplied or approved by Alber GmbH.

Damage resulting from

- using the device improperly
- use by a person who has not been instructed or qualified in the handling of the device
- use that contravenes the instructions and recommendations in this operating manual
- exceeding the technical limits laid down in this operating manual

shall not be deemed by Alber GmbH as part of the company's liability.

Alber GmbH excludes any liability for damage caused

- **by use contrary to proper purpose of the Erivo and its accessory parts;**
- **by the Erivo and its accessory parts being handled by a person who has not been instructed or qualified;**
- **by use of the Erivo and its accessory parts contrary to the instructions and specifically the safety and hazard information in this operating manual;**
- **or by exceeding the technical limits laid down in this operating manual on the part of the user or third parties.**

Before commissioning the Erivo, carefully read all safety and hazard information contained in the individual chapters of this operating manual.

1.5. Signs and symbols

Important tips and information are identified in this operating manual as follows:

Indicates tips and special information.

Warning against possible hazards to your safety and health, and indication of possible injury risks. Warning against possible technical problems or damage.

Observe these indications and warnings at all times to avoid injury to people and damage to the product.

Information in this operating manual, such as "in front", "behind", "left", "right", "forwards", "backwards" etc. refers to the position **from the point of view of the wheelchair user.**

The symbols used on the labels and in some parts of this operating manual are explained in the following.

The Erivo complies with all applicable sections of EN 12184 for electrically powered wheelchairs, as well as the relevant normative references of this standard, in particular ISO 7176-14 and EN 1021-2 (or equivalent ISO 8191-2). It is therefore in compliance with the EU Medical Devices Regulation 2017/745 (MDR) and is classified as a Class I medical device as per said Regulation.

This product features the UKCA mark, in compliance with Part II UK MDR 2002 (as amended) Class I.

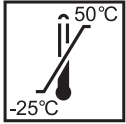
Medical device

For information on how to dispose of the Erivo and its components, please see chapter 12.5.

Protect the device from moisture.

Indicates the temperature range in which the device can be used.

Observe the operating manual!

This operating manual contains instructions, information and warnings relating to operation of the Erivo and to charging the batteries. They are to be read and be given due consideration prior to commissioning the Erivo or charging it for the first time.

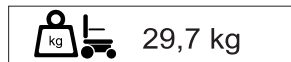
4

Specification of the manufacturing date on the system label (see also chapter 18).


Name and address of the device manufacturer (see the back of this operating manual).

29,7 kg

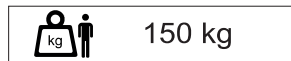Weight of the Erivo R10 (29,7 kg).


150 kg

Maximum weight of person that the Erivo can accept as its load capacity (150 kg).

6°

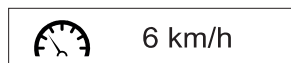Maximum slope (6° / 10,5%).

6 km/h

Maximum speed (6 km/h).

For explanations of the stickers found on the Erivo, see chapter 18.

1.6. Permissible conditions of use/locations of operation

- Observe the information regarding the permissible conditions of use in this operating manual.
- It is imperative that any limits regarding the operation of the wheelchair (e.g. maximum gradeability, maximum permissible height of obstacles, maximum user weight etc.) be observed.
- The Erivo may only be operated at temperatures between -25 °C and +50 °C. Therefore, do not expose the Erivo to any heat sources (such as intense sunlight) as this may cause surfaces to reach high temperatures.
- Avoid using the wheelchair on soft ground (e.g. loose chipping, sand, mud, snow, ice or deep puddles), and avoid using it in rain, thick fog, storms and poor visibility.
- Take particular care to observe the safety instructions and hazard information provided from chapter 3 onwards.

When not in use, do not expose the Erivo to strong sunlight for long periods of time. This could cause the motor to heat up and, in extreme cases, prevent it from operating at full power. Plastic parts also age quicker under intense sunlight.

The execution of “wheelie” movements (anti-tippers removed from wheelchair, drive wheels on the ground, front wheels (castors) up in the air) is not permitted. Alber GmbH accepts no liability for accidents resulting from this or other such actions.

1.7. Standard scope of delivery

- Erivo wheelchair
- Control unit
- Battery pack
- Charger with adapter
- This operating manual

1.8. Optional accessories

Please refer to the Alber homepage for information about the available accessories.

We recommend using only original accessories supplied by Alber.

The Erivo is designed in such a way that it performs best when using Alber original accessories. Alber GmbH shall not be liable for damage to the product or accidents (e.g. fires) caused by a malfunction of non-original accessories or spare parts. The warranty does not cover repairs for damage caused by malfunction of non-original accessories. Alber GmbH will, however, carry out such repairs for you at a charge.

1.9. The main elements at a glance

(please fold out overview drawing in document cover)

Components and operating elements

Control unit	1
Control unit fixation	2
Armrest, left/right	3
Backrest	4
Backrest fixation	5
Seat cushion	6
Battery pack	7
Lever for the folding mechanism	8
Battery pack fixation	9
Lever for operating modes	10
Valve (on both drive wheels)	11
Foot support	12

Battery pack

Battery pack	7
Button for system restart	13
Socket for the system’s power supply	14
Charger socket	15

Charger

Charger	21
LED display	22
Charging plug	23
Adapter	24
Mains plug	25

Control unit

Control unit	1
Joystick	16
Button for warning signal (horn)	17
Combined On/Off button with speed control	18
LED display	19
Charger socket (on the underside)	20

The diagrams contained in this operating manual may deviate from the product supplied to you.

2. Commissioning

On delivery, your Erivo is ready for use with all standard operating components and any optional accessories you may have ordered.

On delivery of your Erivo, your specialist dealer will show you how to operate the system and any accessories you have ordered. You will also be given this operating manual which, in addition to the technical information, also contains important driving instructions. Read through the operating manual carefully, particularly all safety and hazard information.

The first trip with the Erivo may not take place until after this operating manual has been read.

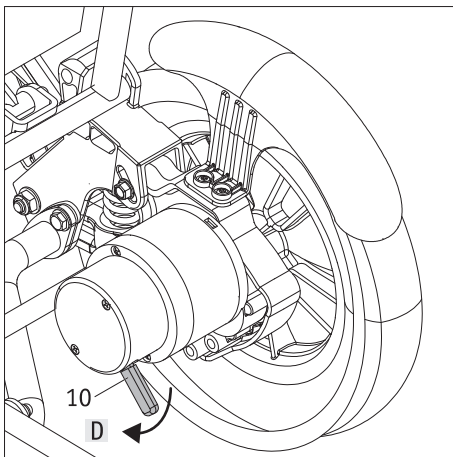
Before the Erivo is used for the first time, its battery pack [7] must be fully charged (see chapter 4.3.1 and 4.3.2).

The Erivo is quickly brought to a standstill in the event of a system failure or other serious errors in the system, as this is the safe state in accordance with the standard. The wheelchair user must be able, both physically and in terms of his reaction time, to counteract the braking acceleration forces that occur in this regard. If the wheelchair user is not able to do this, straps must be worn for trips with the Erivo.

Similarly, do not smoke while using the wheelchair. Burning cigarettes could accidentally fall onto the seat cushion [6] and possibly cause it to melt or set it on fire.

2.1. Switching on the system

Before you begin a trip, you should ensure that the battery pack has been fully charged (see chapter 4.3.1 and 4.3.2). If this is the case, proceed as follows:

- Push the lever [10] on both motors down to position D. This will put the Erivo into electric motor operation.
- Switch on the control unit [1] (see chapter 5.1).
- Begin your trip.
- If it is not possible to start your trip due to an error or a fault, this will be indicated in the control unit by the switch [18] and the LED display [19] (see chapter 6).

In electric motor operation, the wheels are self-locking once the system has been switched off (see chapter 5.2) and thus perform the same function as a parking brake.

If you are not yet familiar with the Erivo and its travel characteristics, you should start by reducing the maximum speed to a minimum using the switch [18] (see chapter 5.5).

When travelling with the Erivo, you must observe the safety instructions and hazard information in chapter 3.

The Erivo may only be operated at temperatures between -25 °C and +50 °C. Do not expose the Erivo to any heat sources (such as intense sunlight) as this may cause surfaces to reach high temperatures.

When not in use, do not expose the Erivo to strong sunlight for long periods of time. This could cause the motor to heat up and, in extreme cases, prevent it from operating at full power. Plastic parts also age quicker under intense sunlight.

The Erivo cannot be pushed manually in electric motor operation.

2.2. Range

For every Erivo user, the range of the system is one of the areas of greatest interest. In general, it is possible to say that this is roughly 10 km. These are ideal values and relate to flat, paved terrain. Deviations may occur due to topographical conditions, the ambient temperature, the road surface, the tyre pressure of the front wheels, the frame geometry, the weight of the wheelchair, the weight of the user and the wheel tyres.

2.3. Switching off the system

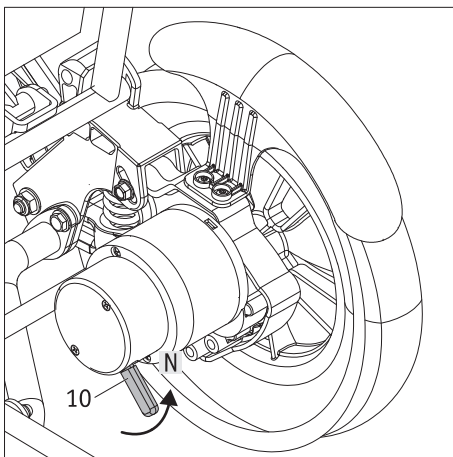
Once you have completed your trip and will not be using your wheelchair for a prolonged period of time, you should always shut down the Erivo. Firstly, this saves energy and secondly, the Erivo is not inadvertently deployed by accidentally touching the joystick. Please read chapter 5.2 in this regard.

2.4. Travelling in push mode

In addition to electric motor operation, the Erivo wheels can also be operated manually by an attendant for push mode.

Push mode is activated as follows:

- Switch off the system at the control unit (see chapter 5.2).
- Pull the lever [10] on both motors upwards to position N. This will disengage the wheels and put them into push mode.
- If you mistakenly try to control the wheels in manual mode, an error message will be shown on the control unit (see chapter 6).

Both wheels are in freewheeling mode in push mode. This means that there is no braking function. In case of a prolonged standstill, the wheels should therefore be switched to electric motor operation in order to prevent the Erivo rolling away unintentionally.

For reasons of safety, the drive wheels must only be disengaged from motorised mode and set to push mode while stationary and on a flat surface. The drive wheel must be load-free when being disengaged. To do so, switch off the Erivo at the control unit. Actuating the clutch with a load or while travelling can damage the internal mechanics and is therefore not covered by the warranty. Disengaging on an incline can lead to a dangerous situation as the Erivo either switches directly or subsequently to freewheeling mode and, in the worst case, could therefore start moving in an uncontrolled manner, i.e. it could rotate and/or roll downhill. Disengagement should only be performed on an incline in case of emergency and only if an accompanying person is present and capable of securing the wheelchair manually.

3. Safety instructions and hazard information when using the Erivo

3.1. General information

- The Erivo wheels are controlled via the control unit joystick (see chapter 5.3). If the system is switched on, each actuation of the joystick is converted into a travel command. This relates to moving both forwards and backwards as well as accelerating and braking. For this reason, switch off the control unit in the event of a prolonged standstill, in order to prevent the wheelchair rolling away inadvertently due to touching the joystick accidentally.
- We recommend that you have a brief user training session when you first receive your Erivo. This will allow you to get used to the drive and everything you can do with it.
- When transferring into or out of the Erivo, its drive wheels must be switched to electric motor operation to prevent the Erivo inadvertently rolling away.

3.2. Information about user training

- The safety and well-being of the user is the top priority. To ensure this, it is essential to understand the Erivo's travel characteristics. Your specialist dealer or Alber agent will help you in this regard with a complimentary device training session.
- Observe the information, safety instructions and hazard information provided in this operating manual.
- Start your first travel attempts with caution and start your user training on level ground.
- Take your user training in areas without any obstacles.
- Always adapt your speed to external conditions so that you are able to stop your Erivo and circumvent any obstacles that may suddenly appear, for example.
- Never immediately push the joystick to its maximum position when starting off. This may result in the wheelchair moving uncontrollably (risk of accident).
- Move the joystick gently and without sudden movements. Avoid suddenly moving the joystick to its maximum position, especially in potentially hazardous situations that you wish to avoid. Brake the Erivo carefully in advance until you come to a stop.
- When the joystick is released, your wheelchair will be braked gently. If you need to brake suddenly (immediate stop) then deflect the joystick briefly in the opposite direction to which you are travelling and then release it.
- Only brake the Erivo using the joystick control unit. Never grip the tyres "for support".
- Before using the Erivo to navigate slopes or inclines, you should be able to operate the device safely on level ground.
- Always navigate inclines with a fully charged battery pack.
- Never travel on slopes with a fully charged battery pack, as doing so could damage the battery pack and the motors.
- You should exercise particular caution when travelling on slopes of any kind.
- Travel on slopes at reduced speed, particularly so that you are able to avoid any obstacles that appear suddenly or stop the wheelchair in time.
- Never travel across steep slopes or similar conditions. Unintentionally shifting the centre of gravity could result in the wheelchair tipping over on its side.
- When navigating inclined kerbs or in similar circumstances, you may need to countersteer a little. Never travel over minor obstacles such as kerbs by moving parallel to them, move at a right angle to the respective obstacle. This means that both wheels travel over the obstacle at the same time rather than being offset. Use a low speed that is appropriate for the obstacle.
- Never remove the anti-tippers, not even when traversing larger obstacles.
- When travelling on public roads you must observe the provisions of the road traffic regulations. Your wheelchair is equipped with the additional equipment specified in this manual.

Exercise caution when travelling downhill with a fully charged battery pack.

Travelling at high speeds with a fully charged battery pack may lead to a reduction in speed or the system automatically switching off. Therefore, reduce your speed.

Negotiate obstacles (e.g. kerbs) as slowly and carefully as possible. The maximum permitted obstacle height is 3 cm. It is up to you to decide whether to enlist the help of another person.

In the event of a technical problem or error message, immediately contact your specialist dealer.

Observe the safety instructions and hazard information below.

3.3. Safety instructions

- The system must be switched off at the control unit before starting work on the wheelchair.
- For safety reasons, you should not stop on inclines or slopes. This is to prevent the wheelchair from rolling away inadvertently.
- In the (unlikely) event of the battery pack overheating or catching on fire, you must ensure that the battery pack does not come into contact with water or other liquids. The only suitable extinguishing agent recommended by the battery manufacturers is sand.
- Using the Erivo as a vehicle seat is not permitted.

Before beginning your trip:

- Perform a functional check. In this regard, see the table in Appendix A – Checks and inspections.
- Wheelchair use is not permitted if the anti-tippers have been removed.
- Before each trip, check the condition of the Erivo wheels. If the tyres have reached their wear limit (the tyre tread is less than 1 mm deep), the Erivo must no longer be used.
- Check the air pressure of the Erivo drive wheels at regular intervals. The table in chapter 17 contains information about the correct air pressure. This information can also be found on the outside of the tyres. Insufficient air pressure can severely affect the handling and range.
- Both Erivo drive wheels should always have the same air pressure.
- The Erivo must not be used in combination with a wheelchair towing device or hand bike, or other pulling devices.

Observe the following points when on a trip with the Erivo:

- Before using the Erivo to navigate slopes or inclines, you should be able to operate the device safely on level ground.
- The maximum permissible slope specified by the wheelchair manufacturer must not be exceeded.
- Always begin your trips at the lowest speed and then slowly increase your speed according to external conditions.
- Be very careful when approaching steps and drops.
- If the Erivo is switched on, each touch of the joystick is converted into a travel command. When stopping or waiting in potentially hazardous areas (e.g. while waiting at a traffic light, on slopes and inclines or ramps of any type), do not touch the control unit.
- Never reach into the wheel when the Erivo is switched on.
- Do not insert or throw objects of any kind into an Erivo wheel, regardless of whether or not it is switched on.
- Never attach objects of any kind to the Erivo wheels. This could cause damage.
- When travelling on pavements, keep a safe distance (preferably the width of a wheelchair) from the kerb.
- Avoid using the wheelchair on soft ground (e.g. loose chipping, sand, mud, snow, ice or deep puddles).
- Never leave the Erivo unattended, even if it is switched off.
- The Erivo may be adversely affected by strong electromagnetic fields.
- Under certain circumstances, the Erivo may interfere with other electrical equipment such as anti-shoplifting barriers in department stores.
- You must not use the Erivo on escalators or moving walkways.
- When moving at maximum speed, do not suddenly change your direction of travel to the left or right, as this could cause the wheelchair to tip sideways.
- Reduce your speed when turning a corner.
- If you intend to stop on a slope or incline with your Erivo, your wheelchair must be parked perpendicular to said slope or incline.
- Never travel across slopes.
- Never reach into or touch the Erivo drive wheels or front wheels when travelling.
- Apply the Erivo brake carefully and adjust your speed accordingly (i.e. not suddenly).
- When travelling in a vehicle, always sit in the vehicle's installed seats with the appropriate restraint systems. Failure to comply with this guideline may result in injury to you and your co-passengers in case of an accident.
- When travelling in a vehicle, secure your Erivo according to the current applicable statutory requirements and legislation.
- Using the Erivo as a vehicle seat is not permitted.
- When navigating public roads and pavements in the Federal Republic of Germany, the provisions of the German Road Traffic Regulations and the Road Traffic Licensing Regulation are to be observed at all times. In other countries, the locally applicable national provisions are to be observed at all times.
- The Erivo is only intended for transporting persons with restricted mobility and must not be used for any other purpose, e.g. as a plaything for children, for transporting children or for transporting goods.
- Operation of the Erivo is prohibited in the vicinity of strong magnetic fields, such as those induced by holding solenoids, transformers, tomographs, etc.
- Avoid using the Erivo in adverse conditions, such as storms, hail and high undergrowth.

Observe the following points after your trip with the Erivo:

- Switch off the Erivo when not in use to prevent inadvertently triggering travel impulses when the joystick is touched and to prevent the self-discharge of the battery pack.
- If possible, recharge the battery pack of your Erivo after each trip.

3.4. Special safety instructions for travelling uphill, downhill and on ramps

In order to achieve maximum wheelchair stability and thus maximum user safety, we recommend the following sitting posture when travelling:

Travelling on level ground and on slopes

Travelling on inclines and on ramps

Maximum incline/slope: 6 degrees
Maximum user weight: 150 kg

It is also imperative that the following points be observed:

- The maximum user weight is 150 kg.
- It is only permitted to travel on slopes and inclines that do not exceed 6 degrees. Never travel on slopes that exceed the nominal incline.
- Travel on slopes and inclines, particularly ramps, only at a low and situation-appropriate speed.
- Avoid navigating tight corners on slopes and inclines.
- In particular, do not attempt any turns on ramps.
- Avoid navigating long ramps and especially uneven terrain, in order to reduce the risk of a motor defect.
- The ability of your Erivo to navigate inclines is affected by, among other factors, your weight, speed and angle of approach to the incline.
- Note the sitting posture shown above, especially the sitting posture applicable to inclines and ramps.

3.5. Obstacles

- The maximum permitted obstacle height is 3 cm.
- Attention! Move towards an obstacle slowly and carefully until the Erivo wheels touch the obstacle. Once you reach this point, negotiate the obstacle carefully. It is up to you to decide whether to enlist the help of another person.

3.6. Hazardous areas and hazardous situations

- The wheelchair user must decide in each case whether it is safe to navigate the intended route based on factors such as their skill level and their physical strength.
- Before undertaking a trip, check the Erivo wheels for damaged or deflated tyres. Also, check the charge status of the battery pack and that the warning signal (horn) on the control unit is working correctly.
- These safety checks and the personal experience of the wheelchair user are of particular importance in or near the following hazardous areas, the navigation of which is left to the judgement and risk of the Erivo user:
 - Quay walls, berths and marinas, paths and areas near water, bridges and dykes without railings
 - Narrow footpaths, gradients (e.g. ramps and driveways), narrow paths along slopes, mountain paths
 - Narrow and/or steep/inclined paths along main and side roads or near precipices
 - Paths covered with leaves, snow or ice
 - Ramps and lifting equipment on vehicles
 - Paved paths/roads

When cornering or turning the wheelchair on slopes or inclines, the centre of gravity of the wheelchair may be shifted leading to a risk of the wheelchair tipping over to the side. As such, proceed with extreme caution and reduce your speed.

Particular caution is required when crossing roads, crossroads and level crossings. Never cross rail tracks in roads or at level crossings by travelling parallel to them, as the castors of the wheelchair could get stuck in the rails.

Proceed with particular caution when navigating ramps and lifting equipment on vehicles. Before lifting or lowering the wheelchair on the ramp or lifting equipment, park the Erivo at a safe distance from the edge and switch it off.

Never use your wheelchair directly at the edge or close to the edge of a ramp or lifting device, instead leave a sufficient distance.

If the ground is wet, tyre grip could be poor, leading to an increased risk of slipping. Adapt your user behaviour accordingly and never travel with worn tyres.

In order to prevent technical damage, the Erivo should not be used if it is raining or during snowfall.

Please note that when travelling downhill the brake path of the Erivo may be significantly longer than on level ground depending on the speed and gradient. Therefore, adjust your speed accordingly.

Never travel on slopes with a fully charged battery pack, as doing so could damage the battery pack and the motors.

When navigating paved paths/roads, it is necessary to exercise greater caution, as the castors of the wheelchair could get jammed in the grooves on these surfaces.

In the event of fire or smoke, wheelchair users are particularly at risk of being fatally or seriously injured as they are unable to move away from the source of the fire or smoke. Lit matches, lighters and cigarettes can cause an open flame around the wheelchair or on clothing. Avoid using or storing the wheelchair near open flames or flammable products and do not smoke while using the wheelchair.

Using the Erivo as a vehicle seat is not permitted.

The execution of "wheelie" movements (anti-tippers removed from the Erivo, Erivo wheels on the ground, front wheels (castors) of the wheelchair up in the air) is not permitted. Alber GmbH accepts no liability for accidents resulting from such actions.

3.7. Electromagnetic interference (EMI)

Every device with electrical components can cause interference and thus interfere with the operating ability of other devices. Operation of your Erivo could therefore be disrupted by radio and television transmitters, amateur radio stations, two-way radios, and mobile phones. In extreme cases, this could result in the Erivo releasing its brakes and moving of its own accord. There could also be permanent damage to the Erivo's control system.

Your Erivo can withstand electromagnetic interference of up to 20 V/m.

EMI warnings

- We generally recommend refraining from trips in the vicinity of electrical interference fields.
- Do not use any portable transceivers, such as radios or mobile phones, while the Erivo is switched on.
Recommendation: Switch off the devices completely prior to a trip and do not switch them back on until you have completed your trip. The Erivo should be switched off completely prior to commissioning the devices.
- Pay attention to nearby powerful transmitters, such as radio or TV transmitters, and keep your distance from them.
- Switch off the Erivo completely if you notice any unintended, independent movement of the Erivo during electric motor operation. Do not use it again until you are in an environment where there is little or no electromagnetic interference.
- Please note that the addition of accessories and components, as well as any modification, could make the Erivo more susceptible to electromagnetic interference.

In rare cases, operation of the Erivo may interfere with other electrical devices such as anti-shoplifting barriers in department stores.

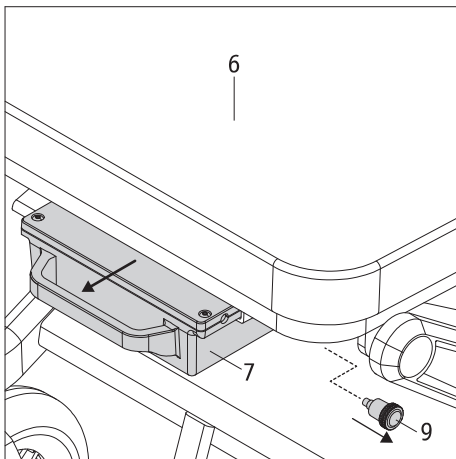
4. Battery pack

Before commissioning the Erivo and before charging the battery pack, read and observe the general information and instructions and the safety instructions and warnings in chapters 4.5 to 4.9. Failure to comply with the safety instructions and warnings could damage the product or result in electric shock, fire and/or serious injuries.

Alber GmbH cannot be held liable for damage caused by non-compliance with the general information and safety instructions.


Only use the battery pack enclosed with the Erivo, or a battery pack supplied by Alber. The warranty/guarantee is rendered void if a battery pack from a third-party manufacturer is used. Moreover, no liability for damage is accepted.

4.1. Removing the battery pack

Generally speaking, the battery pack [7] remains on your wheelchair at all times. If it does need to be removed (e.g. for transport purposes), please proceed as follows:

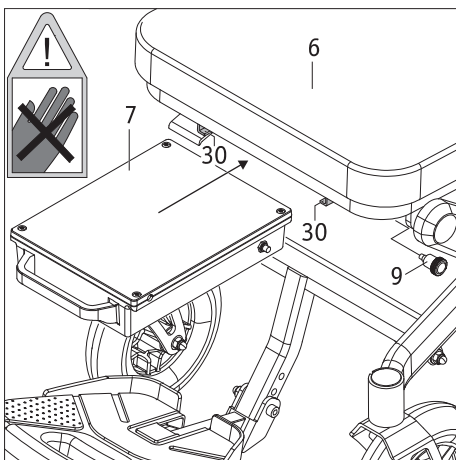- Switch off the system at the control unit (see chapter 5.2).
- Pull the fixation [9] on the left-hand side below the front end of the seat out slightly and, at the same time, use the handle to remove the battery pack [7] from the wheelchair.

4.2. Installing the battery pack

If the battery pack has been removed from the wheelchair, you have to re-install it in the wheelchair in order to commission the Erivo in electric motor mode.

- Ensure that there are no foreign particles (especially metallic objects) in the socket [14]. Clean the socket if necessary.
- Carefully insert the battery pack [7] into the two guide rails [30] as shown in the diagram opposite. In doing so, the fixation [9] does not need to be pulled outwards.
- Push the battery pack back as far as it will go (Attention: Risk of crushing). The battery pack is automatically locked by the fixation [9] as soon as it is in the correct position. A clearly audible “click” will be heard in this regard.

4.3. Charging the battery pack

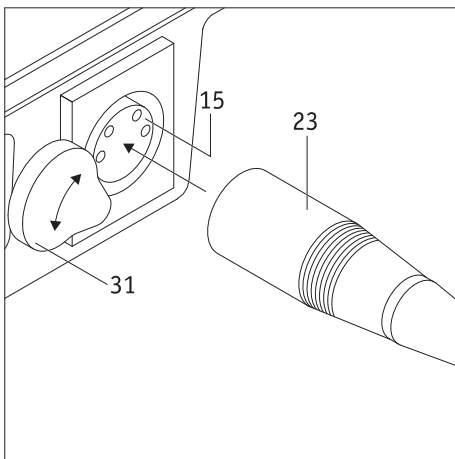
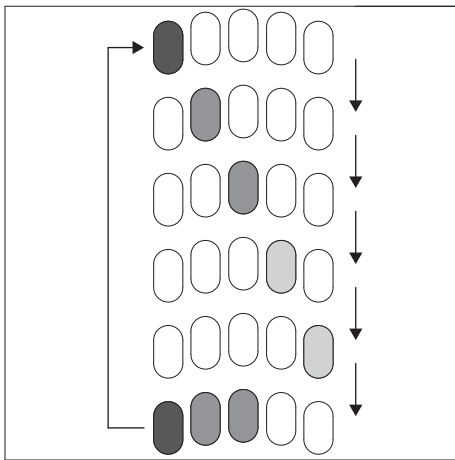
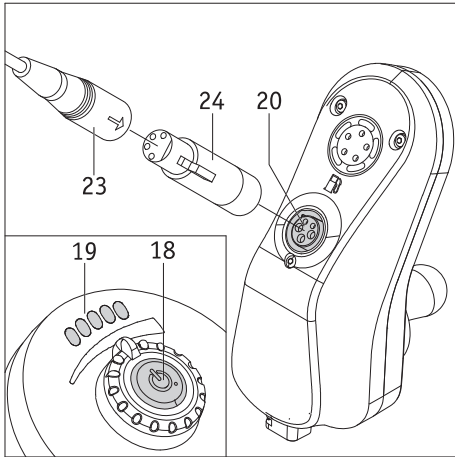
The battery pack for your Erivo should be recharged after any extensive use of your Erivo.

This can be done via the control unit (chapter 4.3.1) or directly on the battery pack (chapter 4.3.2) that has been removed from the wheelchair.

The charging process can take up to roughly 7 hours. The charging progress can only be monitored when charging via the control unit.

Do not interrupt the charging process until the battery pack is fully charged. The LED display [22] on the charger will then light up green.

The battery pack should always be fully charged.

4.3.1. Charging the battery pack on the control unit

We recommend charging the battery pack on the control unit, as you can monitor the approximate charge status via the LED display [19].

- Switch off the Erivo (see chapter 5.2).
- Switch the Erivo to electric motor operation (see chapter 2.1) in order to prevent the wheelchair rolling away inadvertently during the charging process.
- Plug the charging plug [23] into the adapter [24].
- Insert the adapter [24] into the charger socket [20] of the control unit.
- Connect the charger to a socket.
- Charging commences. The LED [22] on the charger lights up orange.
- Press the button [18] if you wish to know the charge status. This will briefly light up green and then flash red 6 times in succession.

At the same time, the LED display [19] indicates the current charge status. In this context, all LEDs will light up briefly in succession, from left to right. Thereafter, the approximate charge status is indicated by the illumination of one or more LEDs. In the diagram opposite, this would be (as an example) three illuminated LEDs in the bottom row.

If the charging process is fully completed, the LED [22] of the charger will light up green.

- Next, first disconnect the charger from the mains supply and then remove the adapter [24] and charging plug [23] from the charger socket [20].
- If the charger has been disconnected from the mains supply but is still connected to the control unit, electric motor operation of the wheelchair will be blocked.

The control unit is deactivated during the charging process. Consequently, electric motor operation of the wheelchair is not possible.

4.3.2. Charging the battery pack in its charger socket

- Remove the battery pack [7] from the wheelchair (see chapter 4.1).
- Swing the cover [31] to one side.
- Insert the charging plug [23] (without the adapter [24]) into the charger socket [15] on the back of the battery pack.
- Connect the charger to a socket. Charging commences. The LED [22] on the charger lights up orange.
- The LED [22] of the charger lights up in green if the charging process is complete.
- Next, first disconnect the charger from the main supply, then remove the adapter [24] and the charging plug [23] from the charger socket [20] and swivel the cover [31] over the charger socket.

When charging the battery pack, only a charger supplied by Alber may be used.

A different charger must not be used under any circumstances.

Before starting the charging process, switch off the system and secure the wheelchair to prevent it inadvertently rolling away by activating electric motor operation.

The charger must not be left permanently attached to the Erivo. Disconnect it from the mains socket first before disconnecting it from the battery pack. Do so as soon as the charging process is complete.

Always recharge the battery pack fully. Only interrupt the charging process in an emergency. The duration of the charging process depends on the residual capacity of the battery pack.

4.4. Capacity display

The battery pack [7] required to operate the Erivo is fitted as standard underneath the wheelchair seat. It does not have a capacity display. The battery pack capacity available is therefore indicated by the LEDs [19] on the control unit. This comprises 5 individual LEDs, which are arranged as follows from left to right: 1 x red, 2 x orange, 2 x green.

When the control unit is switched on, these LEDs light up and indicate the battery pack charge status to you as follows:

LED display	Status in operation	Status when charging
	The battery pack is fully charged.	The battery pack has been fully charged.
	The battery still has 60% to 80% residual capacity.	The battery is between 60% and 80% charged
	The battery still has 40% to 60% residual capacity.	The battery is between 40% and 60% charged
	The battery still has 20% to 40% residual capacity. The battery should be fully charged at this point, especially if an extensive trip is planned.	The battery is between 20% and 40% charged
	Only minimal residual capacity remaining The battery needs to be charged immediately.	The battery is charged to 20%

There is an error if the LEDs indicate signals that deviate from the table. For details, see chapter 6.2.

4.5. General information about charging the battery pack

- Do not charge the battery pack in the presence or vicinity of flammable liquids or gases.
- In the (unlikely) event of the battery pack overheating or catching on fire, you must ensure that the battery does not come into contact with water or other liquids. The only suitable extinguishing agent recommended by the battery manufacturers is sand.
- Do not carry out the charging process in rooms where moisture may affect the battery pack.
- Carry out the charging process with the wheelchair in a location that has at least twice the space required by the wheelchair itself and that provides sufficient ventilation, in order to avoid the danger of flammable gas accumulating.
- Explosive gases may form during charging. Keep the wheelchair and charger away from sources of ignition, such as flames and sparks.
- Never charge the battery pack at temperatures under 0 °C or over 40 °C.
- Never use an extension cable to connect the charging plug [23] or the adapter [24] to the socket [14] of the battery or the charger socket [20] of the control unit.

4.6. Safety instructions for the charger and the charging process

- Switch off the system before starting the charging process. If the battery pack is being charged directly on the wheelchair, the wheelchair is to be secured to prevent it inadvertently rolling away by activating electric motor operation (see chapter 2.1).
- Only use the enclosed charger to charge the battery pack. The charging process ends automatically when the battery pack is fully charged. As such, overcharging is impossible.
- Never use any charger other than the one supplied by Alber.
- Do not charge any other batteries with the charger. Only charge the Erivo battery pack.
- The charger must not be left permanently attached to the Erivo. Check the charging progress from time to time. Disconnect the charger from the mains socket first before disconnecting it from the battery pack. Do so as soon as the charging process is complete.
- Do not expose the charger to moisture of any kind (water, rain water, snow, etc.) during charging.
- Be aware of condensation. If the charger is brought from a cold area into a warm room, condensation may occur on and in the device. In this case, refrain from using the charger until all condensation has dissipated. Please note that this may take several hours.
- Never carry out the charging process in rooms where moisture may affect the charger.
- Never carry the charger by its power cable or the charger cables.
- Never tug at the power cable to disconnect the charger from the mains socket.
- Lay the mains cable and charger cable so that no-one can tread on them or trip over them. Do not expose the two cables to any other harmful influences or stresses.
- Do not operate the charger if the power cable, the charger cable or the plugs attached to the cables are damaged. Damaged parts must be replaced immediately by a specialist dealer that has been authorised by Alber.
- Do not use or disassemble the charger if it has received a hard blow or was dropped or damaged in some other way. Take the damaged charger to a specialist dealer that has been authorised by Alber to perform repairs.

- The charger must not be used by young children.
- The charger is designed to operate only with mains alternating voltage of 100 V to 240 V.
- Never attempt to dismantle or modify the charger.
- Do not cover the charger during the charging process or place any objects on top of the device.
- Never subject the cable or plug to any pressure. Overstretching or bending the cable, pinching cables between a wall and a window frame or placing heavy objects on a cable or a plug may result in electric shock or fire.
- Never connect the terminals of the plug [23] on the charger cable to any metallic objects.
- Ensure that the mains plug is firmly inserted in the socket.
- Do not touch the mains plug or the charger cable plug with wet hands.
- Do not use the plug of the charger and/or the mains plug if they are wet or dirty. Before plugging them in, clean the plugs using a dry cloth.
- Once the charging process is complete, first disconnect the charger plug from the socket and then disconnect the charging plug from the socket on the battery pack.
- Ensure that no moisture can get into the socket [14] on the battery after disconnecting the charger.
- Store the charger in a dry, well ventilated environment.
- The charger must not be stored in places where it is exposed to direct heat sources, humidity, water, oil, acids or alkalis, or in places where ozone could potentially be generated.

The chargers supplied by Alber may only be used to charge the battery pack of the Erivo. The chargers must not be used to charge other batteries under any circumstances.

Before and after charging, ensure that the charger adapter [24] and the socket [14] on the battery pack are clean and that there are no metallic particles on them. If there are any metallic particles, these must be removed with a clean, dry cloth.

4.7. Safety instructions for the battery pack

- Before the Erivo is used for the first time, the battery pack should be fully charged.
- The Erivo must only be used at temperatures between -25 °C and +50 °C.
- Never charge the battery pack at temperatures under 0 °C or over 40 °C.
- The battery pack must not be exposed to heat (e.g. radiators) or fire. External heat exposure can lead to the battery pack exploding.
- In the (unlikely) event of the battery pack overheating or catching on fire, you must ensure that the battery pack does not come into contact with water or other liquids. The only suitable extinguishing agent that is recommended by the cell manufacturers is sand.
- Your Erivo consumes energy each time you use it. Therefore, if possible, fully charge the battery pack after every use.
- The battery pack housing must not be opened or taken apart. Improper opening or wilful destruction carries a danger of serious injury. In addition to this, opening the battery will void any warranty claims.
- Never connect the battery pack socket [14] to metallic objects, and ensure that the contacts do not come into contact with metallic objects (for example with metal filings).
- If the socket [14] is dirty, clean it with a clean, dry cloth.
- Never immerse the battery pack in water.
- If damaged or defective, the battery pack must be checked. Please contact your specialist dealer to learn about the proper procedure that must be used for returns and repairs (see also chapter 4.9).
- If the battery pack is defective or damaged, you must not use the Erivo under any circumstances. Contact your specialist dealer about a repair.
- Always ensure that the battery pack is kept clean and dry.

4.8. Storing the battery pack

- The service life of a battery pack is partly dependent on its storage location. For this reason, do not leave the battery pack or the Erivo wheels in hot locations for prolonged periods. In particular, use the boot of a car parked in the sun only for transport – never as a storage location.
- Store the battery pack in a cool, dry and well ventilated environment where it is protected against damage and unauthorised access.
- To achieve the optimum battery pack service life, it should be stored at a temperature of between -25 °C and 50 °C, an air pressure of between 500 and 1060 hPa, and a humidity of between 45 and 85 percent. Under these conditions, the battery pack can be stored for 6 months.
- The battery pack is to be fully charged prior to storage.
- If the battery pack is stored for longer than 6 months, it also needs to be fully charged every 6 months.
- Do not expose the battery pack to moisture of any kind during storage (water, rainwater, snow, etc.).
- Prior to storage, charge the battery pack and check its charge status every month. If necessary, charge the battery pack if the charge status drops to below 50 percent.

4.9. Safety instructions and precautions on how to transport and ship the battery pack

- Lithium-ion cells are used in the Erivo battery pack. Transport and shipping of the battery pack are, therefore, subject to all applicable statutory requirements, which must be strictly observed. For example, a defective battery pack is generally not allowed to be transported on planes, neither as hand luggage nor as checked luggage.
- If your battery pack is defective, take it personally to your specialist dealer as shipping lithium-ion batteries by post or other carriers is subject to strict rules and regulations. Once again, we recommend that you contact your specialist dealer in advance.
- As transport regulations may change annually, we strongly recommend that you consult your travel operator, airline or shipping line before you set out on your trip in order to learn about the current applicable regulations.

If you require a packaging box for transport purposes that complies with the statutory provisions, you can request this from your specialist dealer.

Discuss the necessary measures with your specialist dealer before sending the battery pack.

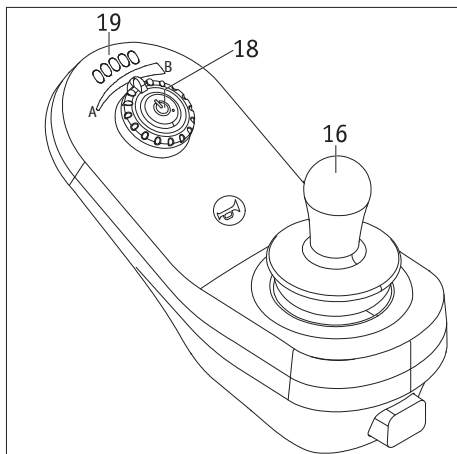
4.10. Important information about air transport

The technical design of the Erivo, and specifically the lithium-ion battery contained therein, complies with the applicable standards, directives, and laws. It meets the requirements of the International Air Transport Association (IATA) for air transport. We confirm this annually in the form of our transport certificate. This is prepared on the basis of the new IATA Dangerous Goods Regulations, which are not usually published until mid-December of each year. The current certificate is available for download on the Alber homepage: (<https://www.alber.de/de/hilfe-service/alber-service/reisen-mit-alber-produkten/>). We would be happy to send it to you on request.

However, when planning a trip, please contact your travel agent or the airlines to discuss taking the Erivo with you on any flight you may book. At the end of the day, the decision to allow or refuse the Erivo on board is made at the pilot's discretion. Neither Alber GmbH nor their specialist dealers are able to influence the pilot's decision.

When travelling, please also note the applicable regulations for transporting lithium-ion batteries in the countries you are visiting.

5. Control unit

5.1. Switching on the system

If the two drive wheels have been correctly switched to electric motor operation as described in chapter 2.1, you can switch on the system using the control unit.

It is imperative that you do not touch the joystick [16] when switching on the system. If you do, you will get an error message.

- Press the On/Off button [18] briefly. The system is switched on, the On/Off button [18] lights up green and the LEDs on the display [19] indicate the capacity of the battery pack (see chapter 4.4).
- If you are not yet familiar with the Erivo and its travel characteristics, you should start by reducing the maximum speed to a minimum (position A) using the button [18] (see chapter 5.5).
- You can now use the joystick [21] to set the two Erivo wheels in motion. However, make sure you read the safety instructions and hazard information in chapter 3 beforehand.
- If the wheels do not move when switched on, the system will automatically switch itself off after 10 minutes.

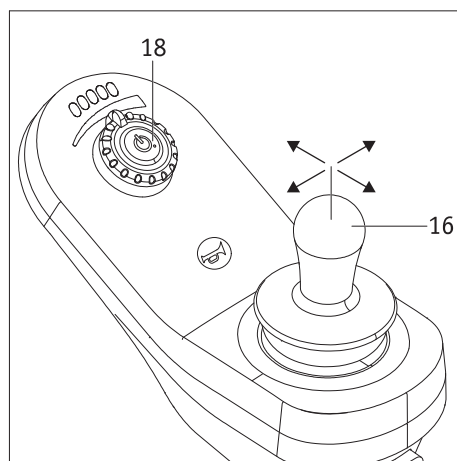
There could be an error present if it is not possible to start a trip after you have switched on the system. Chapter 6 contains a list of these with error codes.

5.2. Switching off the system

Once you have completed your trip and you will not be using your wheelchair for a prolonged period of time, you should always switch off the Erivo. Firstly, this saves energy and secondly, the Erivo is not inadvertently deployed by accidentally touching the joystick.

- Press the button [18]. The control unit is switched off; the LED display [19] is no longer illuminated.

5.3. Using the joystick to travel

The Erivo is moved using the joystick [16], by means of which the user can determine the speed and the direction of travel.

The maximum speed to be reached is initially preset using the button [18]. Even deflecting the joystick slightly in the desired direction of travel will then start travel at low speed. This increases, the further the joystick is deflected. The maximum speed is achieved by fully deflecting the joystick. When travelling in reverse, the speed is automatically reduced for safety reasons.

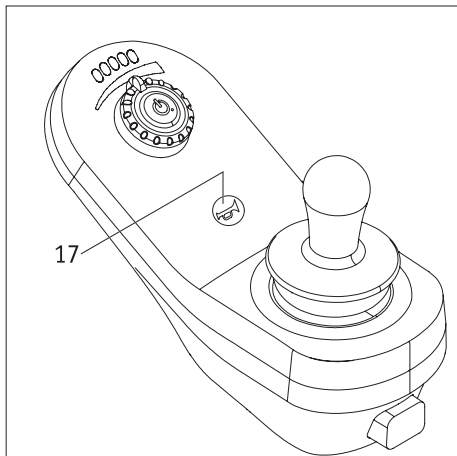
Start your first travelling attempts at a low speed in order to accustom yourself to the travel characteristics of the Erivo.

Adapt the speed to the respective travel situations. Reduce the maximum speed, particularly in difficult travel conditions and in indoor use to prevent accidents.


Do not use the control unit for support as doing so could damage it. Always use the armrest on the wheelchair for support, for assistance with standing up or to adjust your seating position.

5.4. Warning signal

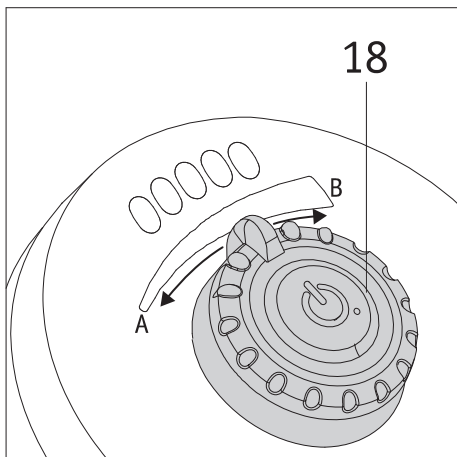Pressing the button [17] will sound a warning signal on the control unit.

5.5. Limiting the speed

The minimum speed of your Erivo is 1.95 km/h, which can be increased in increments and to a maximum speed of 6 km/h if the joystick [16] is fully deflected. For different travel situations, for example indoors, the maximum speed can also be limited on an individual basis. Use the button [18] to select the maximum speed you require. The speed to be reached is reduced (minimum 1.95 km/h) in the direction of position A and it is increased (maximum 6.0 km) in the direction of position B. The maximum speed that you specify in this way is achieved as soon as you fully deflect the joystick [16]. When travelling in reverse, the speed is automatically reduced to 50 percent for safety reasons.

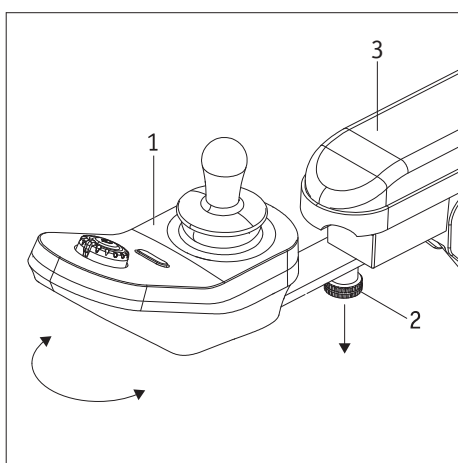
The maximum speed can be set when the wheelchair is at a standstill and when you are travelling.

Start your first travelling attempts at a low speed in order to accustom yourself to the travel characteristics of the Erivo.

Adapt the speed to the respective travel situations. Reduce the maximum speed, particularly in difficult travel conditions and in indoor use to prevent accidents.

5.6. Swivelling the control unit

The control unit [1] can be swivelled, for example, to allow it to be moved closer to a table. To do so, pull down the fixation [2] and, at the same time, swivel the control unit into the desired position. Please note that the control unit is now no longer fixed in place on the armrest [3], but can be moved freely.

If the control unit is swivelled back into its standard position (see diagram opposite), it will automatically be secured on the armrest [3] by the fixation [2] and can no longer be moved.

If the control unit is not in its standard position, only minor travel corrections may be made at the lowest speed; no longer trips may be undertaken with the Erivo.

6. Warnings and error messages

6.1. Error display on the On/Off button [18]

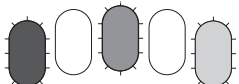
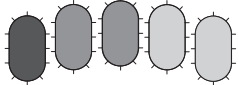
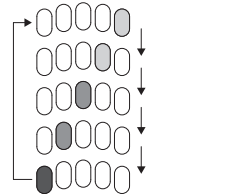
If an error occurs when the system is switched on, the LEDs on the LED display [19] light up in succession from left to right.


The error that has occurred is indicated by the On/Off button [18], which flashes red. The number of consecutive flashes means as follows:

Number of signals	Error	Remedy
1	Joystick error	Check whether the joystick was outside of its mid-position when the system was switched on. Switch off the Erivo, ensure that the joystick is in its mid-position, then switch the Erivo back on. If the error continues to be displayed, please contact your specialist dealer.
2	Network or configuration error	Please contact your specialist dealer.
3	Left-hand motor error	Please contact your specialist dealer.
4	Right-hand motor error	Please contact your specialist dealer.
5	The left-hand motor is in push mode	Switch the left-hand motor to electric motor operation (see chapter 2.1)
6	Both motors or only the right-hand motor are/is in push mode	Switch the right-hand motor to electric motor operation (see chapter 2.1)
7	Module error	Please contact your specialist dealer.

6.2. Error display by the LEDs [19]

In addition to the error messages listed in chapter 6.1, other errors are indicated by the LED display [19] as follows:

LED signals	Error	Remedy
	All LEDs are illuminated; the two green LEDs are flashing. High voltage warning	The battery voltage level has increased. Please contact your specialist dealer.
	The red LED flashes. Low voltage warning	Charge the battery immediately.
	The red LED is flashing and a warning signal sounds every 10 seconds. Cut-off voltage	Charge the battery immediately.
	The left, middle and right LEDs flash. Unlocking the system	The system is locked. Switch the Erivo off and on again. Ensure that the joystick [16] is in its mid-position.
	All LEDs flash. Safety function to prevent the Erivo inadvertently moving off (caused by the deflected joystick) when the system is switched on.	The joystick [16] is not located in its mid-position. Switch the Erivo off and on again. In doing so, ensure that the joystick [16] is in its mid-position.
	All LEDs light up briefly in succession, from left to right in each case. Travelling gear inhibition	Count the number of signals on the On/Off button [18] (see table in chapter 6.1) and remedy the error.

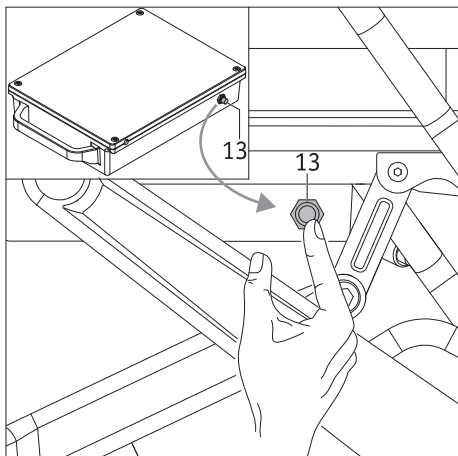
7. Automatic shutdown

7.1. Shutdown due to internal error

In the event of a system malfunction or an overload of the circuit (caused, for example, by travelling with a high load on inclines), the Erivo switches itself off automatically. In order to put the Erivo back into service, you need to restart the system. To do so, press the button [13] on the battery pack located underneath the seat and then press the On/Off button [18] on the control unit.


7.2. Shutdown if not in use

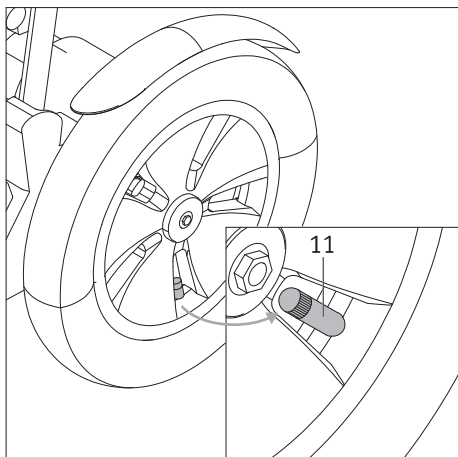If the Erivo is in fact switched on but it is not in use, an automatic shutdown will take place after 10 minutes. To stop such a shutdown, the system needs to be restarted by pressing the On/Off button [18].

8. Air pressure in the tyres

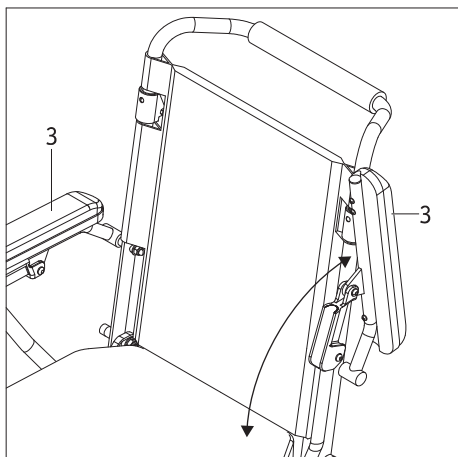The wheels of the Erivo, which are driven by an electric motor, are equipped with pneumatic tyres. Check the air pressure of both tyres (see table in chapter 17) at regular intervals and top up the air if necessary. To do so, carefully pull out the valve [11]. The two castors (front wheels) are made of solid rubber and therefore do not require any air.


9. Adjusting the seating position

- Switch off the Erivo.
- Switch both drive wheels to electric motor operation to ensure that the Erivo cannot roll away inadvertently.
- Both armrests [3] can be swivelled in order to adjust the seating position.
- Use the armrests [3] for support; however, never lean on the control unit for support.




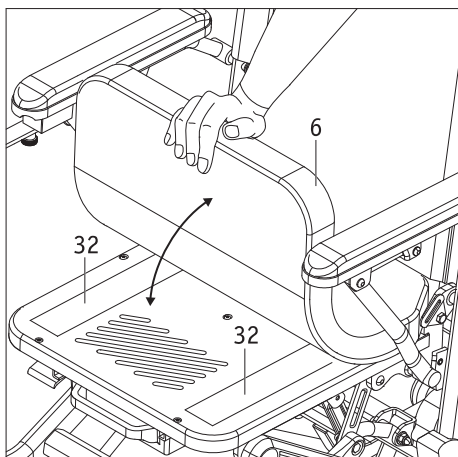
The Erivo must be switched off and both drive wheels must be switched to electric motor operation before you adjust your seating position. This is to prevent the wheelchair from rolling away inadvertently.

10. Removing the seat cushion

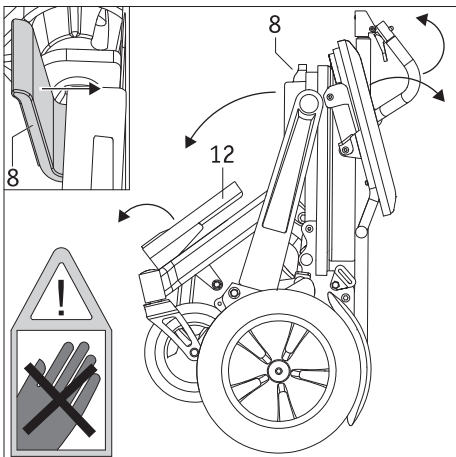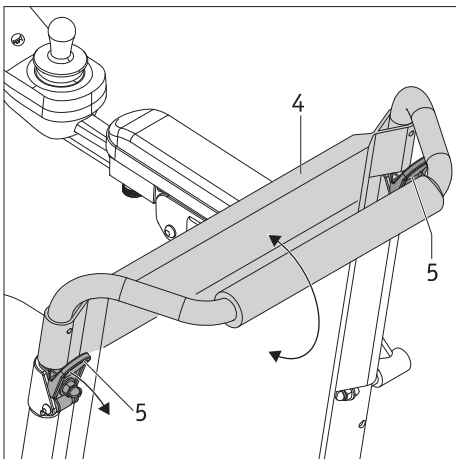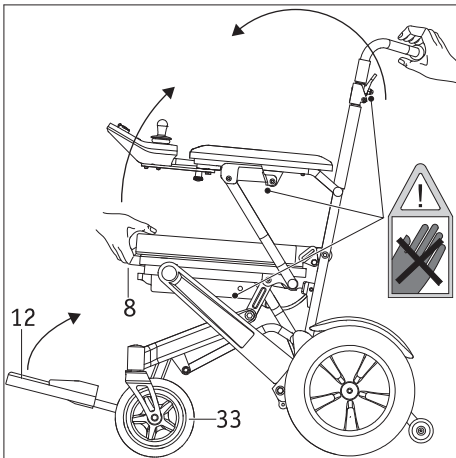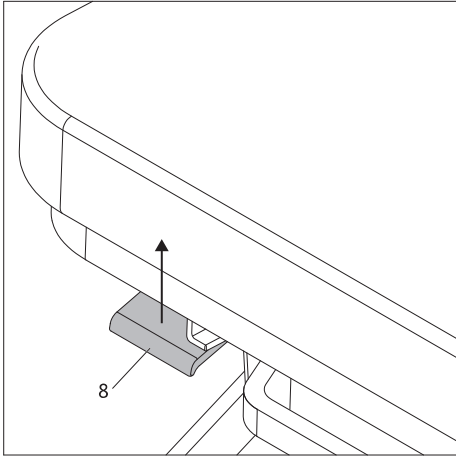The seat cushion [6] is connected to the wheelchair seat by two Velcro straps [32] and can be removed/attached at any time, e.g. for cleaning purposes.

11. Transport

11.1. Folding the wheelchair

Your Erivo can be folded for transport purposes. This is ideally done by an accompanying person.

In this context, the battery pack can be removed or can remain attached to the wheelchair.

- Park the Erivo on level ground; never park it on a slope.
- Switch off the Erivo.
- Pull and **hold** the red lever of the folding mechanism [8] up all the way, at the same time, slide the seat and backrest inwards. **Do not release the lever until the locking pin gets into the hole.**

In doing so, ensure that the two castors [33] (front wheels) do not collide with the components of the wheelchair.

ATTENTION: Risk of crushing!

Please note the safety instructions in this regard.

- The Erivo is fully folded when you hear a clearly audible “click” and the wheelchair can no longer be pulled apart without actuating the lever [8].
- Fold down the foot support [12] to that it rests tightly against the wheelchair.
- Should you wish, the backrest [4] can be folded down at this point. To do so, pull out the two fixations [5] at the same time and swivel the backrest [4] into the desired position.

11.2. Unfolding the wheelchair

- Park the folded-up Erivo on level ground; never park it on a slope.
- Fold down the foot support [12].
- Swivel the backrest back until it is automatically locked by the fixation [5].
- Pull and **hold** the red lever of the folding mechanism [8] up all the way, at the same time, pull the seat and backrest apart. **Do not release the lever until the locking pin gets into the hole.**

In doing so, ensure that the two castors [33] (front wheels) do not collide with the components of the wheelchair.

ATTENTION: Risk of crushing!

Please note the safety instructions in this regard.

- The Erivo is fully folded when you hear a clearly audible “click” and the wheelchair can no longer be pulled apart without actuating the lever [8].

Safety note – Attention: Risk of crushing!

When folding/unfolding the wheelchair, hold it only in the area of the folding mechanism [8] and the backrest, as shown in the diagram. Never reach into the area of any mechanical parts that are pushed together when folding the wheelchair.

Areas in which there is a risk of crushing are marked on the wheelchair with the sticker opposite.

Never hold the wheelchair by the handle of the battery pack when folding/unfolding the wheelchair.

11.3. Safety instructions for loading the wheelchair

11.3.1. Loading without a ramp or lifting platform in a folded state

- Load the Erivo in a folded state with the help of an accompany person.
- Switch off the Erivo (see chapter 5.2).
- Remove the battery pack (see chapter 4.1) in order to prevent the Erivo being switched on unintentionally.
- Activate electric motor operation of the wheels (see chapter 2.1) in order to prevent the wheels rolling away inadvertently.
- Ensure that the Erivo is completely folded together and cannot be pulled apart without actuating the lever [8] (see chapter 11.2).
- When loading or unloading the Erivo, never hold it by its two armrests as the armrests cannot be locked (Risk of crushing!).
- Never hold the Erivo by parts of the leg support, as such parts cannot be locked (Risk of crushing!).

11.3.2. Loading using a ramp or lifting platform in an unfolded state by an accompanying person

- Switch off the Erivo (see chapter 5.2).
- Activate push mode for the wheels (see chapter 2.4).
- When pushing, remain a sufficient distance from the edges of the ramp or lifting platform.
- Only use ramps and lifting platforms with a suitable load-bearing capacity.
- Only load the Erivo using a ramp/lifting platform if you are able to operate it safely in accordance with the manufacturer's specifications.

11.3.3. Loading using a ramp or lifting platform in an unfolded state by the wheelchair user

- Navigate a ramp/lifting platform with extreme care and do so only at the lowest speed.
- With regard to a lifting platform: Keep a sufficient distance away from the edge of the lifting platform.
- Do not attempt any turning manoeuvres on a ramp.
- Only use ramps and lifting platforms with a suitable load-bearing capacity.
- Only load the Erivo using a ramp/lifting platform if you are able to operate it safely in accordance with the manufacturer's specifications.

Observe the safety instructions and hazard information in chapters 3.4 to 3.6

12. Storage, care, re-use, maintenance and disposal

12.1. Storage

If your Erivo is to be stored for an extended period (e.g. several months), please observe the following points:

- Protect the components of the Erivo with film in order to keep moisture out.
- Store the Erivo and its components in a dry room.
- Ensure that no moisture penetrates into or drips onto the Erivo or its components.
- Ensure that the Erivo and its components are not continuously exposed to solar radiation (e.g. through windows).
- Ensure that unauthorised persons (especially children) do not have unsupervised access to this room.
- Please observe the instructions for storing the battery pack in chapter 4.8.
- The Erivo and its components must be cleaned prior to recommissioning. In particular, it is necessary to ensure that there are no metallic particles on/in the plugs [23, 24 and 25] of the charger, the socket [14] of the battery pack or the socket [20] of the control unit.
- Check whether maintenance (see chapter 12.4) is required before recommissioning, and arrange this if necessary.

12.2. Care

Water ingress can permanently damage the drive.

For this reason, never clean the Erivo or its individual components under running water or with a high-pressure cleaner.

Always ensure that no liquids or moisture get into the components, especially the wheels and the battery pack.

Only clean the battery pack with damp (not wet!) cloths. Under no circumstance is liquid to get into the battery pack (Risk of fire!).

Alber GmbH accepts no liability for damage or consequential damage caused by water penetration. Please note that such damage is not covered by our warranty.

Routine cleaning work

- The Erivo is to be switched off and put into electric motor operation prior to cleaning and maintenance work. These measures help prevent the wheelchair rolling away inadvertently.
- Check the Erivo for cleanliness at regular intervals, especially if you have travelled over grass, sand, uneven ground, and potholes.
- Do not expose the Erivo to heat, cold or moisture for prolonged periods of time.
- Keep the control unit clean at all times.
- For all cleaning processes, use cloths lightly moistened with water at most; never use scouring agents or aggressive cleaning agents.
- Never apply liquid cleaners or solvents to the control unit, battery pack, charger or electrical connections.
- Pay special attention to the joystick of the control unit during cleaning work. The joystick must always return to its mid-position. The rubber border must also not be damaged. This is to prevent moisture from penetrating.
- If you notice that parts of the Erivo are no longer fully functional, screws are missing, exposed wires are visible on cables or that there is other externally visible damage, please contact your specialist dealer immediately to arrange a functional check.

Yearly cleaning work

The following components should always be cleaned thoroughly if necessary. In any case, they are to be cleaned at least once a year.

- Armrest: Use a damp cloth and a gentle detergent or 77% alcohol to wipe.
- Seat cushion and backrest: First use a vacuum cleaner, and then a damp cloth and a gentle detergent. Never use abrasive cleaning agents as such agents could damage the seat.
- Control unit: Use a damp cloth to wipe, and then a dry cloth to dry. Never use abrasive cleaning agents.

12.3. Reuse

If your Erivo has been provided to you by your health insurance company and you do not need it any longer, please contact your health insurer, an Alber representative, or your authorised specialist dealer. Your Erivo can then be easily and efficiently reused.

Before the Erivo is reused, it should undergo maintenance.

In addition to the cleaning instructions specified in chapter 12.2, all the externally accessible plastic parts of the Erivo components need to be disinfected before they can be reused. Use only alcohol-based disinfectant wipes approved by the Robert Koch Institute (RKI); see manufacturer's recommendations for application time and concentration.

Example: Bacillol AF, application time 15 minutes.

12.4. Maintenance

In Germany, the maintenance of medical products is regulated by law in the Medical Device Directive (MPBetreibV), Section 7. According to this law, maintenance measures, especially inspections and servicing, are required in order to ensure continuous safe and proper operation of medical devices.

Based on market observations, an interval of two years has proven reasonable for maintenance of our products when used under normal operating conditions. This guideline value of two years may vary depending on the actual use of the product and the behaviour of the user. Checking the degree of use and the user behaviour is the responsibility of the operator. In any case, we strongly recommend that you clarify beforehand to what extent the cost of the maintenance work for our products is covered by your health insurance provider, especially with regard to any existing contracts.

12.5. Disposal

This device, its battery pack and accessories are durable products.

Nevertheless, they may contain substances that are hazardous to the environment if the parts are disposed of in locations (e.g. landfills) that are not intended for the disposal of such materials according to the applicable statutory regulations for the country in question.

The product is labelled (in accordance with the WEEE Directive) with a "crossed-out rubbish bin" symbol, reminding you that it must be recycled.

Please protect the environment and return the product at the end of its service life to your local recycling centre.

As this product is not covered by the regulations of the WEEE Directive in all European countries, please familiarise yourself with the applicable waste disposal regulations in your country. Alternatively, return the Erivo to a specialist Alber dealer or Alber GmbH for proper and environmentally sound disposal.

13. Product safety information

All the components of your Erivo have undergone functional tests and extensive inspections. However, in the event of an unforeseeable impairment in the operation of the Erivo, the following web pages contain prompt and appropriate safety information for customers:

- Homepage of Alber (manufacturer of the Erivo)
- Homepage of the German Federal Institute for Drugs and Medical Devices [BfArM]

If corrective measures are necessary, Alber will notify the specialist dealer, who will then contact you.

14. Product service life

We expect an estimated average service life of five years for this product, provided that the product is used as intended and maintained and serviced in accordance with all the instructions specified in the operating manual. This service life can be exceeded by handling, servicing, maintaining and using the product with care and insofar as there are no technical limitations that arise from further developments in science and technology. However, the service life may also be reduced significantly by excessive or improper use. The service life required by the applicable standards does not constitute an additional warranty.

15. Warranty, guarantee and liability

15.1. Warranty for defects

Alber guarantees that the Erivo is free of defects at the time of transfer. This warranty expires 24 months after the date of delivery of the Erivo.

15.2. Durability warranty

Alber provides a 24-month durability warranty for your Erivo.

The durability warranty does not include:

- Devices whose serial numbers have been tampered with or removed.
- Wear parts such as tyres, operating elements and the spokes.
- Defects caused by normal wear, incorrect handling including non-compliance with the instructions in this operating manual, accidents, reckless damage, damage caused by fire, water, force majeure and other events that are outside the control of Ulrich Alber GmbH.
- Parts that may need to be serviced or replaced as part of normal use (e.g. tyre change).
- Device inspection without detection of a fault or defect.

15.3. Liability

As the manufacturer of the Erivo, Alber GmbH accepts no liability for the safety of the product under the following circumstances:

- If the Erivo is handled improperly
- If the Erivo is not serviced at the prescribed two-year intervals by an authorised specialist dealer or Alber GmbH
- If the Erivo is operated in contravention of the instructions in this operating manual
- If the Erivo is operated with an insufficient battery charge
- The Erivo is repaired or modified by a person not authorised to carry out such work
- If third-party parts are attached to or combined with the Erivo
- If parts of the Erivo are removed

16. Important legal information for the user of this product

Incidents that occur due to product malfunctions and result in serious personal injury are to be reported to the manufacturer and the competent authority of the EU Member State in which the user is established.

With regard to the Federal Republic of Germany, these are:

- Alber GmbH (see the back of this operating manual for details of the address)
- The German Federal Institute for Drugs and Medical Devices (BfArM),
Kurt-Georg-Kiesinger-Allee 3
53175 Bonn
www.bfarm.de

With regard to EU Member States, please notify:

- Your Alber dealership (see the back of this operating manual for details of the address)
- The authorities in your country responsible for incidents involving medical devices

An overview of the competent authorities can be viewed online at: <http://ec.europa.eu/growth/sectors/medical-devices/contacts/>

17. Technical data

Wheel

Range (1):	up to 15 km with standard battery pack 11 Ah (Application class A as per EN 12184) over 25 km with optionally available battery 30.15 Ah (Application class B as per EN 12184)
Nominal gradient:	up to 6° (10.5%)
Speed:	1.95 – 6 km/h Observe the country-specific road traffic legislation.
Rated power of engine:	250 W
Operating voltage:	24 V
Operating temperature:	-25°C bis +50°C
Weight of person:	Maximum 150 kg

Akku-Pack

	Standard Battery pack:	Optional Battery pack:
Cell type:	Lithium-ion cells, leak-proof and maintenance free	
Rated operating capacity:	25.9 V	25.2 V
Rated capacity:	11.0 Ah	30.1 Ah
Rated energy:	284.9 Wh	759.8 Wh
Charging temperature:	0°C to 40°C	0°C to 40°C
Operating temperature:	-25°C to +50°C	-25°C to +50°C
Protection rating:	IP54	IP54

System

Storage temperature:	-40 °C to +65 °C (with the exception of the battery pack)
----------------------	---

Weight of individual parts

Wheelchair without battery pack:	27.2 kg	
Battery pack:	2.4 kg (Standard battery pack)	4.6 kg (Optional battery pack)
Total weight:	29.6 kg	31.8 kg

Wheel tyres

Name	Model and type	Size (inches)	Air pressure in bar / kPa / psi
Drive wheels: Cheng Shin Tire	62 – 203 (12.5" * 12.25") C-628	12.5"	2.8 bar / 280 kPa / 40 psi
Castors: PU front tyres	200 x 50	8"	Puncture-proof

(1) The range varies depending on the terrain travelled on and the prevailing usage conditions. The specified maximum range is valid for optimum travel conditions (level terrain, a newly recharged battery pack, ambient temperature of 20 °C, smooth travel, etc.). At lower temperatures, the range of the Erivo could be lower than the data in the table above.

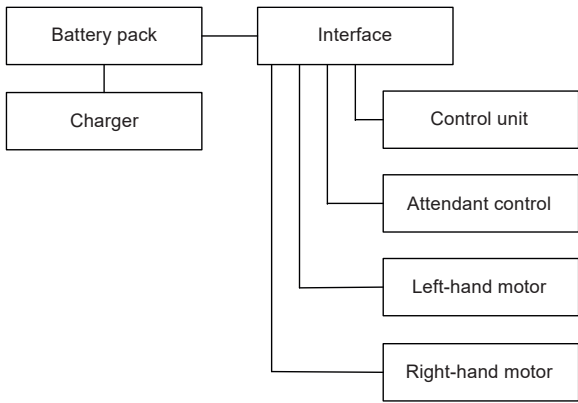
Charger

For technical information and other information on charging, please refer to the separate operating manual for the charger.

Miscellaneous

All Erivo components are protected against corrosion.

Block diagram


CE The Erivo complies with all applicable sections of EN 12184 for electrically powered wheelchairs, as well as the relevant normative references of this standard, in particular ISO 7176-14 and EN 1021-2 (or equivalent ISO 8191-2). It is therefore in compliance with the EU Medical Devices Regulation 2017/745 (MDR) and is classified as a Class I medical device as per said Regulation.

We reserve the right to modify the design and technology of our products to incorporate the latest developments.

18. Labels

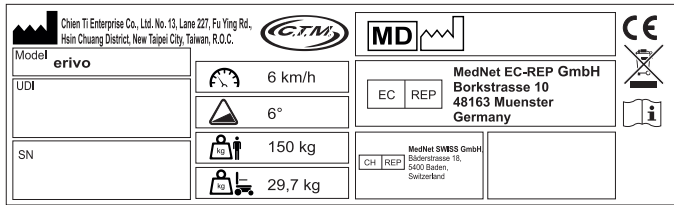There are labels on the Erivo components which include various product details or warnings. The following illustrations are for example purposes only.

System label on the interface

The system label provides information about key technical data of the Erivo and makes it possible, if necessary, to trace the system.

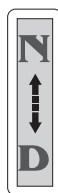
Warning: Hot surface

The surfaces of the drive motors can heat up if they run for a prolonged period of time.

Warning: Risk of crushing.

There is a risk of hands being crushed at the locations marked on the Erivo, especially when folding the wheelchair.

Operating mode

Set the operating mode with the lever [10].

N= push mode

D= electric motor operation

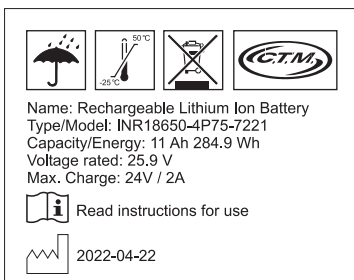
Motor data

The label is located on both motors and provides you with information about its technical data.



Warning

The label is located on the battery pack and contains warnings regarding its handling.

Battery pack data

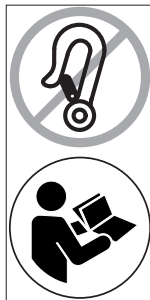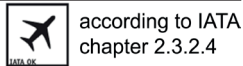The label is located on the top of the battery pack and provides information about its technical data.

Note:

The label is located on the housing of the right-hand motor and informs you that the operating manual states several times that the Erivo must not be used as a vehicle seat.

Appendix

A - Checks and inspections

Before each trip with the Erivo, check its functionality. In this regards, please contact your authorised specialist dealer if malfunctions occur.

Inspection point	Inspection step	Remedy in the event of a malfunction
Control unit	Check whether the LED display [19] lights up after being switched on.	Is the battery fully charged and inserted?
	Check the button [18] and the LED display [19] for any error messages (see chapter 6).	Remedy the errors as per the specifications in this operating manual.
	Check the capacity of the battery pack (see chapter 4.4).	Recharge the battery pack fully before starting your trip.
	Check the button [18] and the joystick [16] for correct operation.	Remedy any errors that occur as per the specifications in this operating manual.
	Check the position of the joystick (when the control unit is switched off!). Ensure that it is not bent out of shape or damaged and that it returns to its mid-position when you release it. Check the rubber sleeve at the base of the joystick for damage. Perform a visual inspection of the sleeve. Do not attempt to repair the joystick.	Contact your specialist dealer in the event of damage or malfunctions.
	Check the horn [17] for correct operation.	Contact your specialist dealer.
Motors	Check the motors for unnatural noises, overheating, and malfunctions.	Contact your specialist dealer.
Brakes	In electric motor operation, check that the brakes hold the Erivo in position when the joystick is released, such that the Erivo cannot roll away.	Contact your specialist dealer.
Operating mode	Check whether the motors can be easily switched to electric motor operation/push mode.	Contact your specialist dealer.
Mechanical system	Check whether the Erivo folded/unfolded without any difficulty	Contact your specialist dealer.
	Check whether the connections of the frame, armrests and foot supports have become deformed or have loosened.	Contact your specialist dealer.
	Check whether all parts are screwed together and whether parts (e.g. screws) are missing	Contact your specialist dealer.
Battery pack	Charging the battery pack on the control unit: Check whether the LED display on the control unit and the LED display on the charger light up after connecting the battery pack.	Check the cable connection between the charger and the control unit. Contact your specialist dealer if you are not able to remedy the error.
	Charging directly on the battery pack: Check whether the LED display on the charger lights up after connecting the battery pack.	Check the cable connection between the charger and the battery pack. Contact your specialist dealer if you are not able to remedy the error.
	The LED illuminated display of the charger does not light up green when it is fully charged.	Contact your specialist dealer.
Wheels and tyres	Check the depth of the tyre tread on all wheels.	Tyres with a tread depth of less than 1 mm must be replaced. Contact your specialist dealer in this regard.
	Check whether the tyres have any cracks or deformation. If there are any foreign bodies in a tyre, these must be removed.	The tyre needs to be replaced if it is cracked or deformed. Contact your specialist dealer in this regard.
	Check the air pressure of the tyres.	Is air constantly escaping from the tyres? Contact your specialist dealer.
	Check whether the wheels turn without wobbling.	Contact your specialist dealer.
Charger	Check the charger for correct operation, especially the LED displays.	Further use of the charger is prohibited if there are any malfunctions. Contact your specialist dealer.

Inspection point	Inspection step	Remedy in the event of a malfunction
Cables	Perform a visual inspection of the cable harnesses of the control system. Ensure they are not frayed or cut and that no wires are exposed.	Contact your specialist dealer if there is a problem with one of the cable harnesses.

Contact your specialist dealer immediately if errors cannot be remedied.

Appendix

B - INVACARE representatives and dealers

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CHIEN TI ENTERPRISE CO. LTD.